

RENACLEAR® DISINFECTANT

For use with the Renaclear® Dialyzer Cleaning System



READ RENACLEAR DIALYZER
CLEANING SYSTEM MACHINE
DIRECTIONS FOR USE PRIOR
TO USING THIS PRODUCT.

Active Ingredients

Hydrogen Peroxide	22.0%
Peroxyacetic Acid	4.5%

Inert Ingredients

	<u>73.5%</u>
Total	100.0%

KEEP OUT OF REACH OF CHILDREN
DANGER

PRECAUTIONARY STATEMENTS

Hazard to humans and domestic animals.

Corrosive: Can cause eye damage and skin irritation. Do not get in eyes, on skin or clothing. Wear safety glasses and rubber gloves when handling. Wash thoroughly after handling. Harmful if swallowed.

STATEMENT OF PRACTICAL TREATMENT

In case of contact with eyes, flush with large amounts of water for at least fifteen (15) minutes. For eyes, get prompt medical attention. If swallowed, drink large amounts of water immediately to dilute. Do not attempt to induce vomiting. Call a physician immediately.

NOTE TO PHYSICIAN: Probable mucosal damage may contraindicate the use of gastric lavage.

For chemical emergency, spill, leak, fire, exposure and accident, call CHEMTREC, day or night (800) 424-9300. In District of Columbia or outside continental U.S.A. call (703) 527-3887.

CAUTION: U.S.A. Federal Law restricts this device to sale by or on the order of a physician.

GENERAL DIRECTIONS FOR USE

1. It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.
2. DO NOT USE AFTER EXPIRATION DATE.
3. Do not allow Renaclear Disinfectant to mix with alkaline substances such as bleach (Sodium hypochlorite).
4. If spilled, flush with large quantities of water.
5. Renaclear Disinfectant is a single use product only intended for use with the Renaclear Dialyzer Cleaning System.
6. Dialyzers require further processing following cleaning. See Renaclear Dialyzer Cleaning System Directions for Use.

DIRECTIONS FOR SANITIZATION

1. Connect the Renaclear Disinfectant container directly to the Renaclear Dialyzer Cleaning System using the Renaclear Disinfectant chemical line. DO NOT dilute Renaclear Disinfectant.
2. Allow Renaclear Disinfectant to dwell in the Renaclear Dialyzer Cleaning System for a minimum of 6 hours for sanitization.
3. For Renaclear Dialyzer Cleaning System installation, low level disinfection will be achieved with a dwell time of 10 minutes.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that:

1. is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or;
2. contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

STORAGE

1. Store in shipping carton.
2. Do not expose to direct sunlight.
3. Maintain temperature below 75°F (24°C).
4. Avoid contact with combustible materials.
5. Avoid contamination from any source, including metals, dust, etc. Such contamination may cause rapid decomposition, generation of large quantities of oxygen gas and high pressures.
6. Store in original closed container.
NEVER TAMPER WITH VENT.

PESTICIDE DISPOSAL

Do not contaminate water, food or feed by storage or disposal. Wastes resulting from the use of this product may be disposed of on-site in a sanitary sewer or at an approved waste disposal facility.

CONTAINER DISPOSAL

Triple rinse container with water. Then offer for reconditioning; or puncture and dispose of in a sanitary landfill; incinerate; or if allowed by state and local authorities dispose of by burning. If burned, stay clear of the smoke.



Minntech Corporation
14605 28th Avenue North
Minneapolis, MN 55447-4822
U.S.A.
Tel: (763) 553-3300
Toll Free: (800) 328-3340
Fax: (763) 553-3387
email: info@minntech.com

Minntech B.V.
Sourthweg 11
6422PC Heerlen
The Netherlands
Tel: (31) 45 5 471 471
Fax: (31) 45 5 429 695
email: info@minntechbv.com

Minntech Japan Corporation
4F Sumitomo Seimei Yochomachi Bldg.
10-10 Yochomachi, Shimjuku-ku
Tokyo 162 - 0055 Japan
Tel: (81) 03 (3225) 8680
Fax: (81) 03 (3225) 8681