

Date: November 14, 2006

<i>Distribution:</i>	
Distributors	✓
Medivators Field Sales	✓
Medivators Customer Support	✓
Medivators Field Service	✓

RE: Scope Buddy™ Programming Times for Olympus, Pentax & Fujinon Brand Endoscopes

The purpose of this document is to provide information concerning the programming time required to flush cleaning and rinse solutions through the endoscope channels during the manual cleaning process.

The times stated assume that the tubes and hookup kits from the SCOPE BUDDY to the endoscope are primed and full of fluid. Program times should be extended by 15 seconds if the tubes are not pre-primed.

The times stated are based on in-house validation tests that were conducted using endoscopes with channels in good condition, free from restrictions and/or defects.

Due to differences in individual endoscope channel performance, users should verify that the solution volumes meet the manual cleaning requirements of the endoscope manufacturer and/or the facility.

1. Programming Time to Flush 100mls through Endoscopes without Elevator Wire Channels

Small Caliber Endoscopes	1 minute 30 seconds
Gastrosopes	1 minute 30 seconds
Duodenoscopes	2 minutes 15 seconds
Colonoscopes	2 minutes 45 seconds
Sigmoidoscopes	2 minutes 45 seconds
Ultrasound Scope	2 minutes 45 seconds
Pentax & Fujinon Forward Water Jet	2 minutes 45 seconds

2. Programming Time to Flush 15mls through the Elevator Wire Channel

Due to the internal structure of Olympus, Fujinon and some Pentax endoscopes fitted with exposed elevator wire channels and the difficulty in passing fluid, these channels must be flushed independently of the other channels.

Olympus	3 minutes
Pentax	4 minutes
Fujinon	6 minutes

If you require any further information regarding this topic, please contact Medivators Customer Support Department on (800) 444 4729 (select option 1).