

SCOPE BUDDY® Flushing Aid Comparison Matrix
(Seconds to flush a minimum of 90mls through each channel)

OLYMPUS® Endoscopes		SCOPE BUDDY Aid	SCOPE BUDDY Aid	ENDO-FLUSH® Aid
Endoscope Type	Endoscope Model	(DSD hookup)	(universal hookup)	
Bronchoscope	BF-10, 20, 30, 40, 60, 160, 180, 200 and 240 series	≥1 min 30 sec	≥1 min 30 sec	Must contact PCI
Colonoscope	All models not listed below	≥2 min 29 sec	≥2 min 29 sec	Not compatible
Colonoscope	CF-140, 160 and 180 series (1 channel models)	≥1 min 30 sec	≥1 min 30 sec	1 min 30 secs
Colonoscope	CF-140, 160 and 180 series (2 channel models)	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Colonoscope	PCF-140, 160 and 180 series	≥1 min 30 sec	≥1 min 30 sec	1 min 30 sec
Colonoscope	CF-1T140L	≥40 sec	≥59 sec	1 min 30 sec
Colonoscope	PCF-130L	≥2 min 7 sec	≥1 min 29 sec	Not compatible
Duodenoscope (excluding elevator wire channel)	All models not listed below	≥2 min 2 sec	≥2 min 2 sec	Not compatible
Duodenoscope (excluding elevator wire channel)	JF-140, TJF-140 and 160 series	≥1 min 30 sec	≥1 min 30 sec	1 min 30 sec
Duodenoscope (excluding elevator wire channel)	JF-140F	≥1 min 30 sec	≥1 min 30 sec	1 min 30 sec
Duodenoscope (excluding elevator wire channel)	TJF-M20	≥46 sec	≥1 min 5 sec	Not compatible
Duodenoscope (elevator wire channel only, 15mls)	All models not listed below	≥1 min 30 sec (using EWC hookup)		1 min 30 secs
Duodenoscope (elevator wire channel only, 15mls)	TJF-M20	≥1 min 26 sec (using EWC hookup)		Not compatible
Gastroscope	All models not listed below	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Gastroscope	GIF-140, 160 and 180 series (1-channel models)	≥1 min 30 sec	≥1 min 30 sec	1 min 30 secs
Gastroscope	GIF-140, 160 and 180 series (2-channel models)	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Gastroscope	SIF-100	≥37 sec	≥50 sec	Not compatible
Gastroscope	GIF-P20	≥48 sec	≥58 sec	Not compatible
Gastroscope	GIF-2T100	≥2 min	≥2 min 15 sec	Not compatible
Choledochoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Cystoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Hysteroscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Laryngoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Pancreatoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Rhinolaryngoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Sigmoidoscope	All models	≥2 min 29 sec	≥2 min 29 sec	Not compatible
Ultrasound Scope	All models	≥2 min 29 sec	≥2 min 29 sec	Not compatible
Uretero-Renofiberscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible

SCOPE BUDDY® Flushing Aid Comparison Matrix
(Seconds to flush a minimum of 100mls through each channel)

FUJIFILM™ Endoscopes		SCOPE BUDDY Aid	SCOPE BUDDY Aid	ENDO-FLUSH® Aid
Endoscope Type	Endoscope Model	(DSD hookup)	(Universal hookup)	
Bronchoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Choledochoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Colonoscope	All models not listed below	≥2 min 45 sec	≥2 min 45 sec	Not compatible
Colonoscope	EC-250HL5	≥2 min 25 sec	≥2 min 13 sec	Not compatible
Cystoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Duodenoscope (excluding elevator wire channel)	All models not listed below	≥2 min 15 sec	≥2 min 15 sec	Not compatible
Duodenoscope (excluding elevator wire channel)	ED-410XU	≥2 min 31 sec	≥63 sec	Not compatible
Duodenoscopes (elevator wire channel only, 15mls)	All models	≥3 min 3 sec (using EWC hookup)		Not compatible
Forward Water Jet	All models not listed below	≥2 min 45 sec (using EWC hookup)		Not compatible
Forward Water Jet	EC-250HL5 (forward water jet only)	≥1 min 45 sec (using EWC hookup)		Not compatible
Gastroscope	All models not listed below	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Gastroscope	EG-450HR	≥1 min 50 sec	≥1 min 20 sec	Not compatible
Hysteroscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Laryngoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Pancreatoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Rhinolaryngoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Sigmoidoscope	All models	≥2 min 45 sec	≥2 min 45 sec	Not compatible
Ultrasound Scope	All models	≥2 min 45 sec	≥2 min 45 sec	Not compatible
Uretero-Renofiberscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
PENTAX® Endoscopes		SCOPE BUDDY Aid	SCOPE BUDDY Aid	ENDO-FLUSH® Aid
Endoscope Type	Endoscope Model	(DSD hookup)	(Universal hookup)	
Bronchoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Choledochoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Colonoscope	All models	≥2 min 45 sec	≥2 min 45 sec	Not compatible
Cystoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Duodenoscope (excluding elevator wire channel)	All models not listed below	≥2 min 15 sec	≥2 min 15 sec	Not compatible
Duodenoscope (excluding elevator wire channel)	ED-3430T	≥1 min 22 sec	≥1 min 29 sec	Not compatible
Duodenoscopes (elevator wire channel only, 15mls)	All models	≥3 min (using EWC hookup)		Not compatible
Forward Water Jet	All models not listed below	≥2 min 45 sec (using EWC hookup)		Not compatible
Forward Water Jet	ES-3870K (forward water jet only)	≥36 sec (using EWC hookup)		Not compatible
Gastroscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Hysteroscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Laryngoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible

Pancreatoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Rhinolaryngoscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible
Sigmoidoscope	All models not listed below	≥2 min 45 sec	≥2 min 45 sec	Not compatible
Sigmoidoscope	ES-3870K	≥1 min 39 sec	≥58 sec	Not compatible
Ultrasound Scope	All models	≥2 min 45 sec	≥2 min 45 sec	Not compatible
Uretero-Renofiberscope	All models	≥1 min 30 sec	≥1 min 30 sec	Not compatible