




1

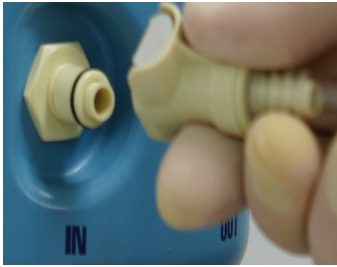

IN Connection

- Connect Fluid Intake Line to “**IN**” port and place filter screen in diluted detergent solution.

2



OUT Connection

- If using DSD Extension Line, connect to “**OUT**” port and to DSD hookup.
- If using Universal Extension Line, connect to “**OUT**” port and to Universal 2, 3 or 4-way Adapter.


3



ATTACH Endoscope

- Connect endoscope to DSD hookup or to Scope Buddy Universal Extension Line.
- Ensure endoscope is fully immersed in fluid filled sink.

4

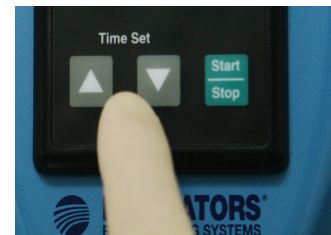

FLUSH Endoscope

- Use ▲ or ▼ arrows to set flush time. Refer to Scope Buddy Rinse Matrix for specific endoscope flush times. Add 5 seconds if tubing unprimed.
- Press “**START/STOP**” to begin cycle. Verify fluid exits all channels.
- Upon completion, tone sounds and display flashes 000.

5



PURGE Endoscope

- Remove Fluid Intake Line from solution and place on counter.
- Press “**START/STOP**” once to reset time; a second time to begin purge cycle.
- Press “**START/STOP**” when fluid no longer exits endoscope.

6

RINSE Endoscope

- Place Fluid Intake Line in clean water.
- Press ▲ or ▼ arrows to reset time and “**START/STOP**” to begin cycle.
- Upon cycle completion, repeat step 5 to purge endoscope.
- Disconnect DSD hookup from DSD Extension Line, or disconnect endoscope from Universal Extension Line.

www.minntech.com/medivators • 1-800-444-4729

Endoscope reprocessing the way it should be

MEDIVATORS