

SCOPE BUDDY™ Flushing Aid Rinse Matrix (For Serial numbers ≥ 201345)

Seconds required to flush a minimum of **90 mls** through each channel

OLYMPUS® Endoscopes		SCOPE BUDDY
Endoscope Type	Endoscope Model	(DSD or universal hookup)
Bronchoscope	All models	≥ 30 sec
Bronchoscope Ultrasound	All models	≥ 60 sec
Colonoscope	All models not listed below	≥ 60 sec
Colonoscope	CF & PCF 140 series and above (1 & 2 channel models)	≥ 30 sec
Colonoscope (using MAJ-855 auxiliary water tube)	CF & PCF 140 series and above (1 & 2 channel models)	≥ 40 sec
Duodenoscope (including elevator wire channel)	All models	≥ 60 sec
Gastroscope (including Enteroscope)	All 1 channel models	≥ 30 sec
Gastroscope (using MAJ-855 auxiliary water tube)	All 1 channel models	≥ 40 sec
Gastroscope	All GIF 2 channel models	≥ 40 sec
Choledochoscope	All models	≥ 30 sec
Cystoscope	All models	≥ 30 sec
Hysteroscope	All models	≥ 30 sec
Laryngoscope	All models	≥ 30 sec
Pancreatoscope	All models	≥ 30 sec
Rhinolaryngoscope	All models	≥ 30 sec
Sigmoidoscope	All models	≥ 30 sec
Ultrasound Scope	All models	≥ 30 sec
Ultrasound Scope (with elevator wire channel)	All models	≥ 60 sec
Uretero-Renofiberscope	All models	≥ 30 sec
Auxiliary water tube (not connected to endoscope)	MAJ-855 (quantity one or two)	≥ 30 sec

* Rinse times represent the grouping of endoscopes. Ensure flushing volumes comply with endoscope manufacturers' guidelines. If tubing is not primed, add an additional 5 seconds to flushing time.

Go to: www.medivators.com for additional copies.

SCOPE BUDDY™ Flushing Aid Rinse Matrix (For Serial numbers ≥ 201345)

Seconds required to flush a minimum of **100 mls** through each channel

FUJIFILM™ Endoscopes		SCOPE BUDDY™
Endoscope Type	Endoscope Model	(DSD or universal hookup)
Bronchoscope	All models	≥ 35 sec
Choledochoscope	All models	≥ 35 sec
Colonoscope	Models 530 Version 1 and below Forward Water Jet channel must be flushed separately.	≥ 1 min 5 sec (for all channels except Forward Water Jet) See "Forward Water Jet" for separate channel flush time.
Colonoscope	Models 530 version 2 and above All channels flushed simultaneously	≥ 1 min 5 sec
Cystoscope	All models	≥ 35 sec
Duodenoscope	All models Elevator Wire Channel must be flushed separately.	≥ 1 min 5 sec (for all channels except Elevator Wire Channel). See "Duodenoscope - (Elevator Wire Channel Only)" for separate flush time.
Duodenoscope (elevator wire channel only, 15mls) using elevator wire channel hookup	All models	≥ 3 min
Forward Water Jet (using elevator wire channel hookup)	All models	≥ 1 min 30 sec
Gastroscope	Models 530 Version 1 and below Forward Water Jet channel must be flushed separately.	≥ 1 min 5 sec (for all channels except Forward Water Jet) See "Forward Water Jet" for separate channel flush time.
Gastroscope	Models 530 version 2 and above All channels flushed simultaneously	≥ 1 min 5 sec
Hysteroscope	All models	≥ 35 sec
Laryngoscope	All models	≥ 35 sec
Pancreatoscope	All models	≥ 35 sec
Rhinolaryngoscope	All models	≥ 35 sec
Sigmoidoscope (See "Forward Water Jet" for separate channel flush time)	Models 530 Version 1 and below Forward Water Jet channel must be flushed separately.	≥ 1 min 5 sec (for all channels except Forward Water Jet) See "Forward Water Jet" for separate channel flush time.
Sigmoidoscope	Models 530 version 2 and above All channels flushed simultaneously	≥ 1 min 5 sec
Ultrasound Scope	All models	≥ 1 min 5 sec
Uretero-Renofiberscope	All models	≥ 35 sec
PENTAX® Endoscopes		SCOPE BUDDY™
Endoscope Type	Endoscope Model	(DSD or universal hookup)
Bronchoscope	All models	≥ 35 sec
Choledochoscope	All models	≥ 35 sec
Colonoscope (including forward water jet w/non-check valve adapter)	All models	≥ 1 min 5 sec
Cystoscope	All models	≥ 35 sec
Duodenoscope (including elevator wire channel)	All models	≥ 2 min
Gastroscope (including forward water jet w/non-check valve adapter)	All models	≥ 35 sec
Hysteroscope	All models	≥ 35 sec
Laryngoscope	All models	≥ 35 sec
Pancreatoscope	All models	≥ 35 sec
Rhinolaryngoscope	All models	≥ 35 sec
Sigmoidoscope (including forward water jet w/non-check valve adapter)	All models	≥ 35 sec
Ultrasound Scope	All models	≥ 35 sec
Uretero-Renofiberscope	All models	≥ 35 sec

* Rinse times represent the grouping of endoscopes. Ensure flushing volumes comply with endoscope manufacturers' guidelines. If tubing is not primed, add an additional 5 seconds to flushing time.