

## SCOPE BUDDY® Flushing Aid Comparison Matrix vs. ENDO-FLUSH® Aid

(For SCOPE BUDDY Flushing Aid serial numbers  $\geq$  201345)

Seconds to flush a minimum of **90mls** through each channel

OLYMPUS® Endoscopes		SCOPE BUDDY Aid	ENDO-FLUSH Aid
Endoscope Type	Endoscope Model	(DSD or universal hookup)	
Bronchoscope	BF-10, 20, 30, 40, 60, 160, 180, 200 and 240 series	$\geq$ 30 sec	<b>Must contact PCI</b>
Colonoscope	All models not listed below	$\geq$ 2 minutes 45 seconds	<b>Not compatible</b>
Colonoscope	CF-140, 160 and 180 series (1 channel models)	$\geq$ 30 sec	1 min 30 sec
Colonoscope	CF-140, 160 and 180 series (2 channel models)	$\geq$ 30 sec	<b>Not compatible</b>
Colonoscope	PCF-140, 160 and 180 series	$\geq$ 30 sec	1 min 30 sec
Colonoscope	CF-1T140L	$\geq$ 30 sec	1 min 30 sec
Duodenoscope (including elevator wire channel)	All models	$\geq$ 60 sec	<b>Not compatible</b>
Duodenoscope (including elevator wire channel)	JF-140, JF-140F, TJF-140 and 160 series	$\geq$ 60 sec	1 min 30 sec
Duodenoscope (elevator wire channel only, 15 mls)	All models not listed below	$\geq$ 60 sec	1 min 30 sec
Duodenoscope (elevator wire channel only, 15 mls)	TJF-M20	$\geq$ 60 sec	<b>Not compatible</b>
Gastroscope	All models not listed below	$\geq$ 30 sec	<b>Not compatible</b>
Gastroscope	GIF-2T100	$\geq$ 60 sec	<b>Not compatible</b>
Gastroscope	GIF-140, 160 and 180 series (1-channel models)	$\geq$ 60 sec	1 min 30 sec
Choledochoscope	All models	$\geq$ 30 sec	<b>Not compatible</b>
Cystoscope	All models	$\geq$ 30 sec	<b>Not compatible</b>
Hysteroscope	All models	$\geq$ 30 sec	<b>Not compatible</b>
Laryngoscope	All models	$\geq$ 30 sec	<b>Not compatible</b>
Pancreatoscope	All models	$\geq$ 30 sec	<b>Not compatible</b>
Rhinolaryngoscope	All models	$\geq$ 30 sec	<b>Not compatible</b>
Sigmoidoscope	All models	$\geq$ 60 sec	<b>Not compatible</b>
Ultrasound Scope	All models	$\geq$ 60 sec	<b>Not compatible</b>
Uretero-Renofiberscope	All models	$\geq$ 30 sec	<b>Not compatible</b>

## SCOPE BUDDY® Flushing Aid Comparison Matrix vs. ENDO-FLUSH® Aid

(For SCOPE BUDDY Flushing Aid serial numbers  $\geq$  201345)

Seconds to flush a minimum of **100mls** through each channel

FUJIFILM™ Endoscopes		SCOPE BUDDY Aid	ENDO-FLUSH Aid
Endoscope Type	Endoscope Model	(DSD or universal hookup)	
Bronchoscope	All models	$\geq$ 35 sec	Not compatible
Choledochoscope	All models	$\geq$ 35 sec	Not compatible
Colonoscope	All models	$\geq$ 65 sec	Not compatible
Cystoscope	All models	$\geq$ 35 sec	Not compatible
Duodenoscope (excluding elevator wire channel)	All models	$\geq$ 65 sec	Not compatible
Duodenoscopes (elevator wire channel only, 15mls)	All models	$\geq$ 3 min (using EWC hookup)	Not compatible
Forward Water Jet	All models	$\geq$ 1 min 30 sec (using EWC hookup)	Not compatible
Gastroscope	All models not listed below	$\geq$ 1 min 30 sec	Not compatible
Gastroscope	EG-450HR	$\geq$ 35 sec	Not compatible
Hysteroscope	All models	$\geq$ 35 sec	Not compatible
Laryngoscope	All models	$\geq$ 35 sec	Not compatible
Pancreatoscope	All models	$\geq$ 35 sec	Not compatible
Rhinolaryngoscope	All models	$\geq$ 35 sec	Not compatible
Sigmoidoscope	All models	$\geq$ 65 sec	Not compatible
Ultrasound Scope	All models	$\geq$ 65 sec	Not compatible
Uretero-Renofiberscope	All models	$\geq$ 35 sec	Not compatible
PENTAX® Endoscopes		SCOPE BUDDY Aid	ENDO-FLUSH Aid
Endoscope Type	Endoscope Model	(DSD or universal hookup)	
Bronchoscope	All models	$\geq$ 35 sec	Not compatible
Choledochoscope	All models	$\geq$ 35 sec	Not compatible
Colonoscope (including forward water jet w/non-check valve adapter)	All models	$\geq$ 65 sec	Not compatible
Cystoscope	All models	$\geq$ 35 sec	Not compatible
Duodenoscope (including elevator wire channel)	All models	$\geq$ 65 sec	Not compatible
Gastroscope (including forward water jet w/non-check valve adapter)	All models	$\geq$ 35 sec	Not compatible
Hysteroscope	All models	$\geq$ 35 sec	Not compatible
Laryngoscope	All models	$\geq$ 35 sec	Not compatible
Pancreatoscope	All models	$\geq$ 35 sec	Not compatible
Rhinolaryngoscope	All models	$\geq$ 35 sec	Not compatible
Sigmoidoscope (including forward water jet w/non-check valve adapter)	All models	$\geq$ 35 sec	Not compatible
Ultrasound Scope	All models	$\geq$ 35 sec	Not compatible
Uretero-Renofiberscope	All models	$\geq$ 35 sec	Not compatible