

Scope Buddy Rinse Comparison Matrix vs. EndoFlush

(For Scope Buddy serial numbers \geq 201345)

Seconds to flush a minimum of **90mls** through each channel

Olympus Endoscopes		Scope Buddy	EndoFlush
Endoscope Type	Endoscope Model	(DSD or universal hookup)	
Bronchoscope	BF-10, 20, 30, 40, 60, 160, 180, 200 and 240 series	\leq 30 sec	Must contact PCI
Colonoscope	All models not listed below	\leq 60 sec	Not compatible
Colonoscope	CF-140, 160 and 180 series (1 channel models)	\leq 30 sec	1 min 30 sec
Colonoscope	CF-140, 160 and 180 series (2 channel models)	\leq 30 sec	Not compatible
Colonoscope	PCF-140, 160 and 180 series	\leq 30 sec	1 min 30 sec
Colonoscope	CF-1T140L	\leq 30 sec	1 min 30 sec
Duodenoscope (including elevator wire channel)	All models	\leq 60 sec	Not compatible
Duodenoscope (including elevator wire channel)	JF-140, JF-140F, TJF-140 and 160 series	\leq 60 sec	1 min 30 sec
Duodenoscope (elevator wire channel only, 15 mls)	All models not listed below	\leq 60 sec	1 min 30 sec
Duodenoscope (elevator wire channel only, 15 mls)	TJF-M20	\leq 60 sec	Not compatible
Gastroscope	All models not listed below	\leq 30 sec	Not compatible
Gastroscope	GIF-2T100	\leq 60 sec	Not compatible
Gastroscope	GIF-140, 160 and 180 series (1-channel models)	\leq 30 sec	1 min 30 sec
Choledochoscope	All models	\leq 30 sec	Not compatible
Cystoscope	All models	\leq 30 sec	Not compatible
Hysteroscope	All models	\leq 30 sec	Not compatible
Laryngoscope	All models	\leq 30 sec	Not compatible
Pancreatoscope	All models	\leq 30 sec	Not compatible
Rhinolaryngoscope	All models	\leq 30 sec	Not compatible
Sigmoidoscope	All models	\leq 60 sec	Not compatible
Ultrasound Scope	All models	\leq 60 sec	Not compatible
Uretero-Renofiberscope	All models	\leq 30 sec	Not compatible

Scope Buddy Rinse Comparison Matrix vs. EndoFlush

(For Scope Buddy serial numbers \geq 201345)

Seconds to flush a minimum of **100mls** through each channel

Fujinon Endoscopes		Scope Buddy	EndoFlush
Endoscope Type	Endoscope Model	(DSD or universal hookup)	
Bronchoscope	All models	\leq 35 sec	Not compatible
Choledochoscope	All models	\leq 35 sec	Not compatible
Colonoscope	All models	\leq 65 sec	Not compatible
Cystoscope	All models	\leq 35 sec	Not compatible
Duodenoscope (excluding elevator wire channel)	All models	\leq 65 sec	Not compatible
Duodenoscopes (elevator wire channel only, 15mls)	All models	\leq 3 min (using EWC hookup)	Not compatible
Forward Water Jet	All models	\leq 1 min 30 sec (using EWC hookup)	Not compatible
Gastroscope	All models not listed below	\leq 1 min 30 sec	Not compatible
Gastroscope	EG-450HR	\leq 35 sec	Not compatible
Hysteroscope	All models	\leq 35 sec	Not compatible
Laryngoscope	All models	\leq 35 sec	Not compatible
Pancreatoscope	All models	\leq 35 sec	Not compatible
Rhinolaryngoscope	All models	\leq 35 sec	Not compatible
Sigmoidoscope	All models	\leq 65 sec	Not compatible
Ultrasound Scope	All models	\leq 65 sec	Not compatible
Uretero-Renofiberscope	All models	\leq 35 sec	Not compatible
Pentax Endoscopes		Scope Buddy	EndoFlush
Endoscope Type	Endoscope Model	(DSD or universal hookup)	
Bronchoscope	All models	\leq 35 sec	Not compatible
Choledochoscope	All models	\leq 35 sec	Not compatible
Colonoscope (including forward water jet w/non-check valve adapter)	All models	\leq 65 sec	Not compatible
Cystoscope	All models	\leq 35 sec	Not compatible
Duodenoscope (including elevator wire channel)	All models	\leq 65 sec	Not compatible
Gastroscope (including forward water jet w/non-check valve adapter)	All models	\leq 35 sec	Not compatible
Hysteroscope	All models	\leq 35 sec	Not compatible
Laryngoscope	All models	\leq 35 sec	Not compatible
Pancreatoscope	All models	\leq 35 sec	Not compatible
Rhinolaryngoscope	All models	\leq 35 sec	Not compatible
Sigmoidoscope (including forward water jet w/non-check valve adapter)	All models	\leq 35 sec	Not compatible
Ultrasound Scope	All models	\leq 35 sec	Not compatible
Uretero-Renofiberscope	All models	\leq 35 sec	Not compatible