

Product Bulletin

For Distribution To:

- International Distributors
- International Managers
- U.S. Field Sales Clinical Specialists
- U.S. Technical & Field Service

Date: April 1, 2015

RE: OLYMPUS® MAJ-855

This bulletin has been released to update the requirements and limitations when reprocessing the OLYMPUS MAJ-855 Auxiliary Flushing Tube in conjunction with MEDIVATORS endoscope reprocessors. The OLYMPUS Reprocessing Manual states that the MAJ-855 is one of the “reusable parts which are normally reprocessed separately from the endoscope”.

The following OLYMPUS 160/180/190 series endoscopes contain auxiliary water channels and may utilize the MAJ-855 Auxiliary Flushing Tube for diagnostic/therapeutic procedures.

160 Series	Endoscopes	180 Series	Endoscopes	190 Series	Endoscopes
CF-Q160L	CF-Q160I	CF-H180AI	CF-H180AL	CF-H190I	GIF-1TH190
CF-Q160S	CF-Q160AL	CF-Q180AI	CF-Q180AL	CF-H190L	GIF-H190
CF-Q160AI	GIF-2T160	PCF-H180AI	PCF-H180AL	CF-HQ190I	GIF-HQ190
GIF-XTQ160	GIF-1TQ160	PCF-Q180AI	PCF-Q180AL	CF-HQ190L	PCF-H190I
					PCF-H190L

Medivators has conducted testing that demonstrates the following:

MAJ-855 Reprocessed separately from an endoscope.

The MAJ-855 can be reprocessed on its own, without connection to the endoscope, in the DSD EDGE® or DSD-201/SSD-102 only, by using the DSD-110-HU0101 hookup. Two MAJ-855's cannot be reprocessed at the same time. The MAJ-855 cannot be reprocessed in ADVANTAGE®, ADVANTAGE PLUS®, DSD-91E, MV-1, CER-1 or MV-2, CER-2, CER OPTIMA® Model 1, CER OPTIMA® Model 2 reprocessors.

MAJ-855 Reprocessed connected to an endoscope.

The MAJ-855 can be connected to an endoscope via a DSD-110-HU0068 or DSD-110-HU0188 hookup in a DSD EDGE or DSD-201/SSD-102 only and will be high level disinfected in conjunction with the endoscope. This reprocessing capability does not apply to the ADVANTAGE, ADVANTAGE PLUS, DSD-91E, MV-1, CER-1 or MV-2, CER-2, CER OPTIMA Model 1, CER OPTIMA Model 2 reprocessors.

Note:

The above-stated reprocessing recommendations are NOT applicable to the ADVANTAGE, ADVANTAGE PLUS, DSD-91, DSD-91E, MV-1, CER-1 or MV-2, CER-2, CER OPTIMA Model 1, CER OPTIMA Model 2 reprocessors. These AERs have not been validated for the reprocessing of the MAJ-855 on its own, or when connected to an endoscope; nor does MEDIVATORS intend to conduct any such validation on these endoscope reprocessor models in the future. Please ensure that this information is consistently communicated to customers/users. Inadequate or inappropriate reprocessing may cause patient injury.

Flushing of MAJ-855 in conjunction with SCOPE BUDDY® flushing aid (serial number ≥ 201345)

Facilities who wish to use SCOPE BUDDY (with serial number ≥ 201345) to flush cleaning solutions and/or rinse water through the MAJ-855 when connected to the endoscope should use forty (40) seconds of flush time. Up to two MAJ-855's can be flushed at the same time when not connected to an endoscope using a time of thirty (30) seconds.



Medivators Inc.
14605 28th Avenue North
Minneapolis, MN
55447-4822 USA
Tel: +1.800.444.4729
Fax: +763.553.3387

Medivators BV
Sourethweg 11
6422PC Heerlen
The Netherlands
Tel: +31.45.5.471.471
Fax: +31.45.5.429.595

Cantel Medical Asia/Pacific Pte. Ltd.
1 A International Business Park
#05-01 Singapore 609933
Tel: +65.6227.9698
Fax: +65.6225.6848

Medivators Inc. Beijing Representative Office
Room 1801, Floor 18th, Tower A,
Beijing Marriott Hotel, Office Building
No. 7, Jianguomen South Avenue,
Dongcheng District, Beijing 100005 China
Tel: +86 10 65204039

Flushing of MAJ-855 in conjunction with SCOPE BUDDY® flushing aid (serial number ≤ 201344)

Facilities who wish to use SCOPE BUDDY (with an earlier serial number < 201344) to flush cleaning solutions and/or rinse water through the MAJ-855 when connected to the endoscope should use ninety (90) seconds of flush time. Up to two MAJ-855's can be flushed at the same time when not connected to an endoscope using a time of ninety (90) seconds.

If you have any questions related to this bulletin, please contact MEDIVATORS Technical Support at 1-800-444-4729.

OLYMPUS® is a registered trademark of Olympus Corporation.

DSD EDGE®, ADVANTAGE®, ADVANTAGE PLUS®, CER OPTIMA® and SCOPE BUDDY® are registered trademarks of Medivators Inc.