



CHECKLIST ADVANTAGE® PLUS

DAILY START UP

TASK	RATIONALE/ACTION
1. Verify machine is on.	The Advantage Plus should not be unplugged during normal use.
2. Open water shut-off valve to the incoming water line.	To provide incoming water flow. Confirm that the incoming water pressure is at least 40 psi.
3. Ensure the Server and LIO (Logic Input / Output) programs are running.	If not, start Server/Services in the MDS/ADV folder first, then LIO.
4. Check all O-rings on connector manifolds are present and undamaged.	Replace any missing O-rings. If damaged, call Medivators Technical Support at 1-800-444-4729.
5. Inspect connector blocks for wear and tear.	Inspection of connectors and tubing for wear and tear will help to prevent alarms and ensure optimal fluid flow through tubing.

DAILY SHUTDOWN

TASK	RATIONALE/ACTION
1. Close water shut-off valve, except when waterline disinfection is being run.	Close water shut-off valve to protect facility and filter housings.
2. Wipe down basins and lids.	Do not use scouring agents.
3. Keep lids open.	Use foot pedal or control panel to open lids.

WEEKLY

TASK	RATIONALE/ACTION
1. Remove basin drain screen and filter with hook; inspect.	Inspect and clean the basin drain screen and filter of any debris.
2. Run waterline disinfection	Using machine disinfection blocks, run waterline disinfect to ensure waterlines are free of microorganisms.
3. Lubricate O-rings with silicon oil	Lubricate channel manifold and hookup O-rings for use and longevity.

MONTHLY

TASK	RATIONALE/ACTION
1. Replace basin drain filter.	Remove basin drain screen and filter with hook. Insert new basin drain filter and reattach basin drain screen.
2. Clean reusable spray arm filter	Unscrew spray arm cap while unit is idle. Remove reusable filter and clean. Put filter back into cap and reattach.

THREE MONTH

TASK	RATIONALE/ACTION
1. Replace 1 micron and .4 micron water filters.	To ensure adequate water flow to the Advantage Plus.

SIX MONTH

TASK	RATIONALE/ACTION
1. Replace 0.1 micron water filter (bacteria-retentive filter) at least every 6 months.	To ensure adequate water flow to the Advantage Plus and prevent microorganism growth in the waterlines.
2. Run Waterline Disinfection after replacing 0.1 micron bacteria-retentive micron filter.	Using machine disinfection blocks, run water line disinfect program, to destroy any possible microorganisms introduced during filter change.
3. Replace air filters x 4 (2 on each side).	Replace air filters matching color on filter to color on connector.
4. Replace vapor management filter (if applicable)	Turn off fan. Open vapor management system lock, remove and replace vapor management filter. Turn fan back on.

ANNUAL

TASK	RATIONALE/ACTION
1. Have Annual Preventive Maintenance (PM) performed by Medivators Field Service Engineer (FSE)	PMs performed by Medivators FSEs provide inspection and replacement of specified components needed for optimal system performance.