

**Active Ingredient:
GLYCINE**

**Net Contents:
10 Oz. Bottle**



NEUTRA-HYDE

**Reorder #
DNS-1002**

Environmentally responsible neutralizer for Glutaraldehyde and OPA

Use one 10 ounce bottle of Neutra-Hyde to neutralize disinfectant as indicated in the chart below.

Disinfectant	# Gallons	Deactivation Time*
2.6% Glutaraldehyde	10	5 minutes
0.55% OPA	5	15 minutes

** Color change will occur during deactivation.*

Sprinkle evenly into solution and replace lid. If using for a spill, sprinkle liberally over spill and refer to your spill response plan for further instructions.

PRECAUTIONARY STATEMENTS:

This product is NOT to be ingested or brought into contact with the skin and eyes. If contact occurs, flush with cold water for fifteen (15) minutes. If ingested, contact a physician for medical attention. Keep away from children. Review and understand manufacturer's MSDS.

Distributed by:

MEDIVATORS® 14605 28th Avenue North
Minneapolis, MN 55447
Phone: 1.800.444.4729