

**PART B** MEDIVATORS®  
**Rapicide PA** READY-TO-USE  
 High-Level Disinfectant & Sterilant

REF ML02-0125

FOR ENDOSCOPE REPROCESSING IN AN AUTOMATED REPROCESSING SYSTEM OR FOR MANUAL DISINFECTION OF MEDICAL DEVICES.

**INGREDIENTS**

Trisodium phosphate... 0.22%  
 Surfactant..... 0.20%  
 Inert Ingredients ..... 99.58%  
 Total ..... 100.00%

**NET CONTENTS:**

**4.775** Liters  
 (1.26 Gallons)



READ DIRECTIONS FOR USE AND SDS FOR THE RAPICIDE PA READY-TO-USE. READ DEVICE MANUFACTURERS REPROCESSING INSTRUCTIONS AND AUTOMATED ENDOSCOPE REPROCESSING SYSTEM DIRECTIONS FOR USE.

**LOT:**



**MFD:**



**EXP:**

KEEP OUT OF REACH OF CHILDREN  
**DANGER**

Date Opened: \_\_\_\_\_


Discard open solution by: \_\_\_\_\_

**HAZARD STATEMENTS** - Causes skin irritation. Causes serious eye irritation.

**PRECAUTIONARY STATEMENTS** - Wash face, hands and any exposed skin thoroughly after handling. Wear protective gloves/ protective clothing/ eye protection/ face protection. IF ON SKIN: Wash with plenty of water. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue Rinsing. If skin irritation occurs: Get medical advice/ attention. If skin irritation or rash occurs: Get medical advice/attention.

**INSTRUCTION FOR USE:** Use this product in a consistent manner with its labeling. Read the DFU prior to using this product. To activate solution pour contents of Part A (225 mL) into bottle contents of Part B. Mix Part A only with Part B. Do NOT use separately.

**STORAGE:** Keep away from children. Keep in a dry place and store upright in original closed shipping carton at room temperature (below 25°C (77°F)). Expiration date applies to properly stored unused product. Do not use after expiration date. The product may be used for 14 days after mixing Part A and Part B. Do not use after 14-day reuse period. **DISPOSAL:** The use solution does not require additional specific safety measures for people or the environment. Dispose of any unused material including the container in accordance with all applicable European and Local regulations. For 24-hour chemical emergency assistance, call CHEMTREC at 1(703) 527-3887. See Safety Data Sheet for additional information.

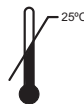

MIX RAPICIDE PA READY-TO-USE PART A WITH BOTTLE CONTENTS OF PART B PRIOR TO USE. DO NOT USE SEPARATELY.

RAPICIDE PA RTU IS REUSABLE FOR UP TO 14 DAYS AFTER MIXING PART A AND PART B PROVIDED CONCENTRATION OF PERACETIC ACID IN THE USE SOLUTION IS VERIFIED USING RAPICIDE PA TEST STRIPS  
 REF ML02-0129 TO BE ABOVE 1000 PPM.

**MEDIVATORS**  
 A Cantel Medical Company

**CE**  
**0050**

Medivators Inc.  
 14605 28th Avenue North  
 Minneapolis, MN 55447 USA  
 Toll Free: +1.800.444.4729

Medivators BV  
 Sourethweg 11  
 6422PC Heerlen  
 The Netherlands  
 Tel: +31.45.5.471.471

EC REP

Cantel Medical Asia/Pacific Pte. Ltd.  
 1A International Business Park  
 #05-01 Singapore 609933  
 Tel: +65.6227.9698

www.medivators.com