

MEDIVATORS®

Rapicide® PA

Sterilant Test Strips

Test Strips
for Peracetic Acid
Level in Sterilant
Solution

REF ML02-0150

NET CONTENTS: 100 strips



CAUTION: READ ENTIRE PACKAGE INSERT FOR IMPORTANT STORAGE / USE INFORMATION. STORE IN A COOL DRY PLACE. DO NOT REUSE TEST STRIP.

DO NOT EXPOSE STRIP TO DIRECT SUNLIGHT. DO NOT REFRIGERATE.



DIRECTIONS FOR USE

RAPICIDE® PA Sterilant Test Strips are used as a chemical indicator after the sterilization cycle to ensure the RAPICIDE PA use solution is above minimum recommended concentration (1700ppm peracetic acid); this ensure the Sterilant was above MRC during the entire sterilization cycle. Refer to the MEDIVATORS ADVANTAGE PLUS®, ADVANTAGE PLUS Pass-Thru or DSD EDGE® User Manuals for instructions on how to take a sample of the disinfectant use solution.

- Dip the strip into the solution for one (1) second.
- Shake off the excess solution.
- Wait for thirty (30) seconds and then compare the strip color to the color chart on the bottle. Read within one (1) minute of testing to determine if the result matches PASS or FAIL.
- Any test strip color other than solid black is a FAIL result.
- Test strips have a four (4) month open bottle shelf life.



PASS

FAIL

Date
Opened

LOT :

EXP :

 **MEDIVATORS**
A Carrel Medical Company

Manufactured in the USA by:

Medivators Inc.
14405 28th Avenue North
Minneapolis, MN 55447 USA
Toll Free: +1.800.864.6739

Medivators Pty
Scouringway 111
6420PC Haverhill
The Netherlands
Tel: +31.46.5.471.471

C.R. Kennedy & Company Pty Ltd
202 Lorraine Street
Port Melbourne 3207
Victoria
Australia
Tel: +61.3.9823.1555



www.medivators.com

© 2016 Medivators Inc. 67200-389/C