

PRECAUTIONARY STATEMENTS:

Use of personal protective equipment including appropriate clothing, gloves, and safety glasses must be worn. Refer to the Material Safety Data Sheet (MSDS) for additional information.

FIRST AID:

In case of contact with eyes, flush with copious amounts of water for 15 minutes. In case of ingestion, DO NOT induce vomiting. Contact CHEMTREC for emergency medical treatment at 1-800-424-9300.

STORAGE & DISPOSAL:

Store at room temperature 15°C-25°C (59°F-77°F). INTERCEPT has a two (2) year shelf life from date of manufacture. Discard unused solution according to local, state and federal regulations. DO NOT reuse container.

DIRECTIONS FOR USE

For cleaning of fully immersible endoscopes, related accessories, surgical instruments, and other apparatus where blood, mucus, protein or other hard to remove soils are encountered, use INTERCEPT at 1/3 oz./gal of water (0.25% use concentration) **with one full stroke of the hand-pump (1 oz.) to three gallons of water.** If visible organic matter is present, use INTERCEPT at 2/3 oz./gal of water (0.5% concentration) **which is two full strokes of the hand-pump (2 oz.) to three gallons of water.** For best manual cleaning results, mix INTERCEPT with cool to warm water (20°C - 35°C) (68°F - 95°F) and ensure a minimum contact time of (1) one minute. Rinse all surfaces and internal channels thoroughly with water. For use in automated washers, follow washer manufacturer's recommendations. **Do not mix INTERCEPT with bleach.**

New Fresh Scent

Intercept™

Next-Generation Detergent

NET CONTENTS
1 U.S. Gallon
(3.8 Liters)

**KEEP OUT OF REACH OF CHILDREN
CAUTION**

CAT NO.
ML02-0106

- Low foaming, neutral pH, bactericidal detergent specifically formulated for use with medical instruments including immersible endoscopes, related accessories, and surgical instruments.
- Patented formula will rapidly penetrate and solubilize blood, proteins, cellular mucosal debris, carbohydrates, lipids and mucopolysaccharides.
- Laboratory proven to effectively remove biological and organic residues from a variety of medical instruments and surfaces.
- Intercept may reduce the film build-up that can occur from the use of non-glutaraldehyde high-level disinfectants on the devices or accessories.
- At the dilution rate of 1/3 oz. per gallon of water, Intercept is the most highly concentrated and most cost effective cleaner for automatic washers, washer sterilizers, ultrasonic washers, and manual cleaning.
- Intended for use in health care settings such as hospitals, surgical centers, neonatal units, medical clinics, emergency medical settings, laboratories, dental offices, ophthalmic offices and veterinary facilities.

endoscope reprocessing the way it should be



READ MSDS FOR ADDITIONAL PRECAUTIONARY STATEMENTS.

LOT:



EXP:

MEDIVATORS®

Manufactured in the U.S.A. by: Minntech Corporation
14605 28th Avenue North
Minneapolis, MN 55447 USA
Phone: (763) 553-3300
Toll Free: (800) 328-3340
Fax: (763) 553-2493
Website: www.minntech.com

U.S. Pat. No. 6,855,678 and foreign counterparts
© 2010, MINNTECH CORPORATION 67199-625/A