



NOTE: The DSD has two disinfectant reservoirs, one for Station A and one for Station B. The SSD only has one reservoir (Station A), so disregard the instructions for Station B.

Remove the Existing Disinfectant from DSD/SSD

1. Dump existing disinfectant from Station A and Station B.
 - a. Press the STATION SELECT key to illuminate both Stations A and B.
 - b. Press the DISINFECTANT DUMP.
 - c. Press the START key and allow the disinfectant dump to complete.
2. Visually ensure that both the A-side reservoir and B-side reservoir are empty.
3. Remove both reservoir tanks and wipe clean with a 1:10 bleach solution using a lint-free cloth.
4. Refit reservoir tanks in the machine.
5. Remove the old disinfectant filters.
6. Install new disinfectant filters.

Rinse the DSD/SSD with water

1. Fill both reservoirs with fresh water and bleed disinfectant pumps.
2. Run two standard cycles with the fresh water.
3. After two cycles dump the water from the reservoirs.
 - a. Press the STATION SELECT key to illuminate both Stations A and B.
 - b. Press the DISINFECTANT DUMP.
 - c. Press the START key and allow the disinfectant dump to complete.
4. Fill both reservoirs with fresh water and bleed disinfectant pumps.
5. Run one standard cycle with the fresh water.
 - a. Select cycle and Press the START key.
6. After one cycle dump the water from the reservoirs.
 - a. Press the STATION SELECT key to illuminate both Stations A and B.
 - b. Press the DISINFECTANT DUMP.
 - c. Press the START key and allow the disinfectant dump to complete.
7. Remove the old disinfectant filters.
8. Install new disinfectant filters.

Fill the DSD/SSD with New Solution

1. Fill both reservoirs with new solution.
2. Wipe down basins with a 1:10 bleach solution using a lint-free cloth.

Refer to the daily preventative maintenance check sheet for daay to day preventative maintenance.