SANIDOSE®

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SANIDOSE®

For use in the control of microbial activity in water lines associated with automated endoscope reprocessors

ACTIVE INGREDIENTS

Hydrogen Peroxide.....	22.0%	NET CONTENTS 5 LITERS
Peroxyacetic Acid	4.5%	READ SANIDOSE LABEL
Other Ingredients.....	73.5%	AND APPLICATION NOTES
Total	100.0%	BEFORE USING PRODUCT

KEEP OUT OF REACH OF CHILDREN DANGER

PRECAUTIONARY STATEMENT

Hazard to humans and domestic pets.

DANGER - Corrosive: Can cause eye damage and skin irritation. Do not get in eyes, or on skin or clothing. Harmful if swallowed. Wear goggles or face shield and rubber gloves when handling. Wash hands thoroughly after handling.

DIRECTIONS FOR USE

Read and follow Medivators SANIDOSE Delivery System package instructions. Connect only to a SANIDOSE Delivery System set to deliver 0.08% product. Determine concentration by following the SANIDOSE Test Strips Instructions for Use. Continuous use of SANIDOSE, when used as directed, eliminates the need to do a daily disinfection of automated endoscope reprocessors such as the ADVANTAGE.

STATEMENT OF PRACTICAL TREATMENT

In case of contact with eyes, flush with large amounts of water for at least fifteen (15) minutes. For eyes, get prompt medical attention. If swallowed, drink large amounts of water immediately to dilute. Do not attempt to induce vomiting. Call a physician immediately.

NOTE TO PHYSICIAN: Probable mucosal damage may contraindicate the use of gastric lavage.

For chemical emergency, spill, leak, fire, exposure and accident, call CHEMTREC, +1-703-527-3887.

STORAGE AND DISPOSAL

- Store in shipping carton.
- Do not expose to direct sunlight.
- Maintain temperature below 75°F (24°C)
- Avoid contact with combustible materials.

SANIDOSE DISPOSAL

Do not contaminate water, food, or feed by storage or disposal. Dispose of this product in accordance with all applicable European and Local regulations.

CONTAINER DISPOSAL

Triple rinse empty container with water. Dispose of this product in accordance with all applicable European and Local regulations.

MEDIVATORS®

	Manufactured in the USA by:		
MINNTECH CORPORATION Minneapolis, MN USA		MINNTECH BV  The Netherlands	MINNTECH JAPAN Tokyo Japan
Toll Free: +1-800-444-4729		Tel: +31 045 5 471471	Phone: +81 03 6226-5885