

SANIDOSE® TEST STRIPS

CATALOG NO ML02-0110 100 STRIPS



CAUTION: READ IMPORTANT STORAGE & USE INFORMATION, STORE COOL AND DRY. Store in a cool dry environment. It is not necessary to refrigerate. Once opened store at room temperature. **Do Not Refrigerate.** Failure to store test strips properly will result in inaccurate readings. When stored and used properly the test strip pad can indicate acceptable Sanidose levels.

INSTRUCTION

- Collect sample of the rinse solution.
- Remove a test strip from the tube and immediately replace the lid.
- Dip the test strip into the solution to be tested for one second, such that the reaction zone is properly wetted.
- Remove the test strip. Gently shake off excess liquid and compare the reaction zone with the color scale between 5 and 10 seconds after exposure.

NOTE: If strip indicates dark blue to brown or green to brown, the concentrations are acceptable.



Endoscope reprocessing the way it should be™

MEDIVATORS®

Manufactured in the USA by:

Minneapolis, MN 55447 USA

Toll Free: 1.800.328.3340

Minnotech B.V.

Tel: +31.455.471.471

Minnotech Asia Pacific Pte Ltd

Tel: +65.6227.9698

67199-342/C



LOT :

EXP: