

**Scope Buddy Rinse Matrix (for serial numbers ≤201344)**

Seconds to flush a minimum of **90mls** through each channel

Olympus Endoscopes Endoscope Type	Endoscope Model	Scope Buddy (DSD hookup)	Scope Buddy (universal hookup)
Bronchoscope	BF-10, 20, 30, 40, 60, 160, 180, 200 and 240 series	≤1 min 30 sec	≤1 min 30 sec
Colonoscope	All models not listed below	≤2 min 29 sec	≤2 min 29 sec
Colonoscope	CF-140, 160 and 180 series (1 channel models)	≤1 min 30 sec	≤1 min 30 sec
Colonoscope	CF-140, 160 and 180 series (2 channel models)	≤1 min 30 sec	≤1 min 30 sec
Colonoscope	PCF-140, 160 and 180 series	≤1 min 30 sec	≤1 min 30 sec
Colonoscope	CF-1T140L	40 sec	59 sec
Colonoscope	PCF-130L	2 min 7 sec	1 min 29 sec
Duodenoscope (excluding elevator wire channel)	All models not listed below	≤2 min 2 sec	≤2 min 2 sec
Duodenoscope (excluding elevator wire channel)	JF-140, TJF-140 and 160 series	≤1 min 30 sec	≤1 min 30 sec
Duodenoscope (excluding elevator wire channel)	JF-140F	1 min 3 sec	1 min 30 sec
Duodenoscope (excluding elevator wire channel)	TJF-M20	46 sec	1 min 5 sec
Duodenoscope (elevator wire channel only, 15mls)	All models not listed below	≤1 min 30 sec (using EWC hookup)	
Duodenoscope (elevator wire channel only, 15mls)	TJF-M20	1 min 26 sec (using EWC hookup)	
Gastroscope	All models not listed below	≤1 min 30 sec	≤1 min 30 sec
Gastroscope	GIF-140, 160 and 180 series (1-channel models)	≤1 min 30 sec	≤1 min 30 sec
Gastroscope	GIF-140, 160 and 180 series (2-channel models)	≤1 min 30 sec	≤1 min 30 sec
Gastroscope	SIF-100	37 sec	50 sec
Gastroscope	GIF-P20	48 sec	58 sec
Gastroscope	GIF-2T100	2 min	2 min 15 sec
Choledochoscope	All models	≤1 min 30 sec	≤1 min 30 sec
Cystoscope	All models	≤1 min 30 sec	≤1 min 30 sec
Hysteroscope	All models	≤1 min 30 sec	≤1 min 30 sec
Laryngoscope	All models	≤1 min 30 sec	≤1 min 30 sec
Pancreatoscope	All models	≤1 min 30 sec	≤1 min 30 sec
Rhinolaryngoscope	All models	≤1 min 30 sec	≤1 min 30 sec
Sigmoidoscope	All models	≤2 min 29 sec	≤2 min 29 sec
Ultrasound Scope (including elevator wire channel)	All models	≤2 min 29 sec	≤2 min 29 sec
Uretero-Renofiberscope	All models	≤1 min 30 sec	≤1 min 30 sec
Auxiliary water tube (connected to endoscope)	MAJ-855 (quantity one)	1 min 30 sec	1 min 30 sec
Auxiliary water tube (not connected to endoscope)	MAJ-855 (quantity two)	1 min 30 sec	1 min 30 sec

\* Rinse times represent the grouping of endoscopes. Rinse times may be reduced by individual flow testing of a specific endoscope. Ensure flushing volumes comply with endoscope manufacturer's guidelines

**Scope Buddy Rinse Comparison Matrix (for serial numbers ≤201344)**

Seconds to flush a minimum of **100mls** through each channel

<b>Fujinon Endoscopes</b>		<b>Scope Buddy</b>	<b>Scope Buddy</b>
<b>Endoscope Type</b>	<b>Endoscope Model</b>	<b>(DSD hookup)</b>	<b>(Universal hookup)</b>
Bronchoscope	All models (including 530)	≤1 min 30 sec	≤1 min 30 sec
Choledochoscope	All models	≤1 min 30 sec	≤1 min 30 sec
Colonoscope	All models not listed below (including 530)	≤2 min 45 sec	≤2 min 45 sec
Colonoscope	EC-250HL5	2 min 25 sec	2 min 13 sec
Cystoscope	All models	≤1 min 30 sec	≤1 min 30 sec
Duodenoscope (excluding elevator wire channel)	All models not listed below (including 530)	≤2 min 15 sec	≤2 min 15 sec
Duodenoscope (excluding elevator wire channel)	ED-410XU	2 min 31 sec	63 sec
Duodenoscopes (elevator wire channel only, 15mls)	All models	≤3 min 3 sec (using EWC hookup)	
Forward Water Jet	All models not listed below	≤2 min 45 sec (using EWC hookup)	
Forward Water Jet	EC-250HL5 (forward water jet only)	≤1 min 45 sec (using EWC hookup)	
Gastroscope	All models not listed below (including 530)	≤1 min 30 sec	≤1 min 30 sec
Gastroscope	EG-450HR	1 min 50 sec	1 min 20 sec
Hysteroscope	All models	≤1 min 30 sec	≤1 min 30 sec
Laryngoscope	All models	≤1 min 30 sec	≤1 min 30 sec
Pancreatoscope	All models	≤1 min 30 sec	≤1 min 30 sec
Rhinolaryngoscope	All models	≤1 min 30 sec	≤1 min 30 sec
Sigmoidoscope	All models	≤2 min 45 sec	≤2 min 45 sec
Ultrasound Scope	All models (including 530)	≤2 min 45 sec	≤2 min 45 sec
Uretero-Renofiberscope	All models	≤1 min 30 sec	≤1 min 30 sec
<b>Pentax Endoscopes</b>		<b>Scope Buddy</b>	<b>Scope Buddy</b>
<b>Endoscope Type</b>	<b>Endoscope Model</b>	<b>(DSD hookup)</b>	<b>(Universal hookup)</b>
Bronchoscope	All models	≤1 min 30 sec	≤1 min 30 sec
Choledochoscope	All models	≤1 min 30 sec	≤1 min 30 sec
Colonoscope	All models	≤2 min 45 sec	≤2 min 45 sec
Cystoscope	All models	≤1 min 30 sec	≤1 min 30 sec
Duodenoscope (excluding elevator wire channel)	All models not listed below	≤2 min 15 sec	≤2 min 15 sec
Duodenoscope (excluding elevator wire channel)	ED-3430T	1 min 22 sec	1 min 29 sec
Duodenoscopes (elevator wire channel only, 15mls)	All models	3 min (using EWC hookup)	
Forward Water Jet	All models not listed below	2 min 45 sec (using EWC hookup)	
Forward Water Jet	ES-3870K (forward water jet only)	36 sec (using EWC hookup)	
Gastroscope	All models	≤1 min 30 sec	≤1 min 30 sec
Hysteroscope	All models	≤1 min 30 sec	≤1 min 30 sec
Laryngoscope	All models	≤1 min 30 sec	≤1 min 30 sec
Pancreatoscope	All models	≤1 min 30 sec	≤1 min 30 sec
Rhinolaryngoscope	All models	≤1 min 30 sec	≤1 min 30 sec
Sigmoidoscope	All models not listed below	≤2 min 45 sec	≤2 min 45 sec
Sigmoidoscope	ES-3870K	1 min 39 sec	58 secs
Ultrasound Scope	All models	≤2 min 45 sec	≤2 min 45 sec
Uretero-Renofiberscope	All models	≤1 min 30 sec	≤1 min 30 sec

\* Rinse times represent the grouping of endoscopes. Rinse times may be reduced by individual flow testing of a specific endoscope. Ensure flushing volumes comply with endoscope manufacturer's guidelines