



# SCOPE BUDDY RINSE MATRIX (For Serial numbers $\geq$ 201345)

Seconds required to flush a minimum of **90 mls** through each channel

Olympus Endoscopes		Scope Buddy
Endoscope Type	Endoscope Model	(DSD or universal hookup)
Bronchoscope	BF-10, 20, 30, 40, 60, 160, 180, 200 and 240 series	$\leq$ 30 sec
Bronchoscope Ultrasound	BFUC-160 and 180 series	$\leq$ 60 sec
Colonoscope	All models not listed below	$\leq$ 60 sec
Colonoscope	CF-140, 160 and 180 series (1 channel models)	$\leq$ 30 sec
Colonoscope	CF-140, 160 and 180 series (2 channel models)	$\leq$ 30 sec
Colonoscope	PCF-140, 160 and 180 series	$\leq$ 30 sec
Colonoscope	CF-11T140L	$\leq$ 30 sec
Colonoscope (using MAJ-855 auxiliary water tube)	CF-140, 160 and 180 series (1 and 2 channel models)	40 sec
Duodenoscope (including elevator wire channel)	All models	$\leq$ 60 sec
Gastroscope (including Enteroscope)	All models not listed below	$\leq$ 30 sec
Gastroscope	All GIF 2 channel models	$\leq$ 40 sec
Choledochoscope	All models	$\leq$ 30 sec
Cystoscope	All models	$\leq$ 30 sec
Hysteroscope	All models	$\leq$ 30 sec
Laryngoscope	All models	$\leq$ 30 sec
Pancreatoscope	All models	$\leq$ 30 sec
Rhinolaryngoscope	All models	$\leq$ 30 sec
Sigmoidoscope	All models	$\leq$ 30 sec
Ultrasound Scope	All models	$\leq$ 30 sec
Ultrasound Scope (including elevator wire channel)	All models	$\leq$ 60 sec
Uretero-Renofiberscope	All models	$\leq$ 30 sec
Auxiliary water tube (not connected to endoscope)	MAJ-855 (quantity one or two)	30 sec

\* Rinse times represent the grouping of endoscopes. Rinse times may be reduced by individual flow testing of a specific endoscope. Ensure flushing volumes comply with endoscope manufacturers' guidelines.



# SCOPE BUDDY RINSE MATRIX (For Serial numbers $\geq$ 201345)

Seconds required to flush a minimum of **100 mls** through each channel

Fujinon Endoscopes		Scope Buddy
Endoscope Type	Endoscope Model	(DSD or universal hookup)
Bronchoscope	All models (including 530)	$\leq$ 35 sec
Choledochoscope	All models	$\leq$ 35 sec
Colonoscope (See "Foward Water Jet" for separate channel flush time)	All models (including 530)	$\leq$ 65 sec
Cystoscope	All models	$\leq$ 35 sec
Duodenoscope (excluding elevator wire channel)	All models (including 530)	$\leq$ 65 sec
Duodenoscope (elevator wire channel only, 15mls) using elevator wire channel hook-up	All models	$\leq$ 3 min
Forward Water Jet (using elevator wire channel hook-up)	All models	$\leq$ 1 min 30 sec
Gastroscope (See "Foward Water Jet" for separate channel flush time)	All models (including 530)	$\leq$ 65 sec
Hysteroscope	All models	$\leq$ 35 sec
Laryngoscope	All models	$\leq$ 35 sec
Pancreatoscope	All models	$\leq$ 35 sec
Rhinolaryngoscope	All models	$\leq$ 35 sec
Sigmoidoscope (See "Foward Water Jet" for separate channel flush time)	All models	$\leq$ 65 sec
Ultrasound Scope	All models (including 530)	$\leq$ 65 sec
Uretero-Renofiberscope	All models	$\leq$ 35 sec
Pentax Endoscopes		Scope Buddy
Endoscope Type	Endoscope Model	(DSD or universal hookup)
Bronchoscope	All models	$\leq$ 35 sec
Choledochoscope	All models	$\leq$ 35 sec
Colonoscope (including forward water jet w/non-check valve adapter)	All models	$\leq$ 65 sec
Cystoscope	All models	$\leq$ 35 sec
Duodenoscope (including elevator wire channel)	All models	$\leq$ 65 sec
Gastroscope (including forward water jet w/non-check valve adapter)	All models	$\leq$ 35 sec
Hysteroscope	All models	$\leq$ 35 sec
Laryngoscope	All models	$\leq$ 35 sec
Pancreatoscope	All models	$\leq$ 35 sec
Rhinolaryngoscope	All models	$\leq$ 35 sec
Sigmoidoscope (including forward water jet w/non-check valve adapter)	All models	$\leq$ 35 sec
Ultrasound Scope	All models	$\leq$ 35 sec
Uretero-Renofiberscope	All models	$\leq$ 35 sec

\* Rinse times represent the grouping of endoscopes. Rinse times may be reduced by individual flow testing of a specific endoscope. Ensure flushing volumes comply with endoscope manufacturers' guidelines.