



Employee will demonstrate competency in using the Medivators Advantage Plus Endoscope Reprocessor.

EMPLOYEE NAME: _____

	CRITICAL SKILLS DEMONSTRATION	DATE REVIEWED	MET	NOT MET	COMMENTS
1.	Uses proper Personal Protective Equipment (PPE).				
2.	Properly places endoscope in Advantage Plus basin.				
3.	Selects correct Hook-up Connector Block for each scope model and can attach each connector correctly.				
4.	Verifies Hook-up Connector Block tubing has no holes/tears/cracks, the connectors function properly and the hook-up has not been modified prior to use.				
5.	Places accessories in mesh bag and positions correctly in Advantage Plus basin.				
6.	Correctly enters ID data (as applicable) for scope #, operator, patient and physician via scanning or manual entry.				
7.	Successfully starts a cycle.				
8.	Verbalizes what each area of the module screen means.				
9.	Tests the disinfectant for Minimum Effective Concentration (MEC) after each use/cycle.				
10.	Correctly presses ENTER if MEC passes, or CANCEL if MEC fails, when Advantage Plus prompts "Collect disinfectant sample and check MEC with test strips".				
11.	Verifies the cycle has completed successfully before removing the scope.				
12.	Verbalizes the function of the control panel buttons.				
13.	Identifies and successfully troubleshoots error messages.				
14.	Successfully replaces disinfectant Part A and Part B bottles and operates "Replace Fluids" utility.				
15.	Successfully reloads alcohol and detergents (if applicable) and operates "Replace Fluids" utility.				
16.	Changes air filters and basin filters at appropriate intervals; documents changes.				
17.	Changes the water pre-filters (1µ and .4µ) and bacteria retentive water filter (.1µ) documents changes.				
18.	Successfully disinfects water lines and verbalizes when to run the waterline disinfection program; documents disinfection.				
19.	Verbalizes where to locate Medivators phone number for technical support, ordering Advantage Plus supplies and service. Explains Diagnostic capability (if applicable).				
20.	Utilizes Quick Start Guide, Filter Log, Hook-up Guide, and User Manual, as appropriate.				

PASS: REDO:

EMPLOYEE SIGNATURE: _____ DATE: _____

REVIEWER'S SIGNATURE: _____ DATE: _____

www.minntech.com/medivators • Toll Free: 1-800-444-4729

Endoscope reprocessing the way it should be

MEDIVATORS