



FILTER PACKAGES

Advantage Complete Filter Package (28 Day HLD Change) - MF01-0061

Filter package includes: Air filter (1/pkg of 8), Disinfectant filter (3/pkg of 12), Basin drain filter (3/pkg of 12), 0.45µ Pre-filter (4), 1µ Pre-filter (4) and 0.2µ Bacterial-retentive absolute filter (2).

Advantage Complete Filter Package (14 Day HLD Change) - MF01-0060

Filter package includes: Air filter (1/pkg of 8), Disinfectant filter (5/pkg of 12), Basin drain filter (5/pkg of 12), 0.45µ Pre-filter (4), 1µ Pre-filter (4) and 0.2µ Bacterial-retentive absolute filter (2).

FILTERS

Disinfectant Filters (12 filters/box) - MF01-0058

Change whenever you are changing disinfectant, one located on each side inside cabinet. One box will last approximately 5 months with 28-day solution (i.e. Rapicide). For 14-day disinfectant, you need to double quantities used.

Air Filter (8 filters/box) - MF01-0028

Change filters on both sides every 6 months. Filter pkg will last approximately 12 months.

Basin Drain Filters (12 filters/box) - MF01-0059

Change whenever you are changing the disinfectant. They are located inside of each basin at the drain. Remove these filters with: **Basin Drain Filter Hook Part # 5-6-170.**

0.2 Micron Absolute Bacteria Water Filter (one each) - MF01-0013

This filter is located inside the Advantage. When looking at the pre-filtration system, it is located on the far left. It needs to be changed every 6 months or sooner if poor water conditions exist. Always keep one on hand and a waterline disinfectant process must be performed when changing this filter.

0.45 Micron Water Filter (one each) - MF01-0015

This filter is located outside the Advantage. When looking at the pre-filtration system, it is the filter in the middle of the pre-filtration system in between the 1 micron and the 0.2 micron. Change is pressure related, water pressure must be 40 psi or above. Always keep one on hand. Change at least every 3 months if pressure remains above 40 psi.

1 Micron Water Filter (one each) - MF01-0014

This filter is located inside the Advantage. When looking at the pre-filtration system, it is located on the far right. Change is pressure related, water pressure must be 40 psi or above. Always keep one on hand. Change at least every 3 months if pressure remains above 40 psi.

Active Vapor Management (Charcoal) Replacement Filter (one each) - MF01-0064

If an Active Vapor Management system (AVM) is installed, it is located behind the machine. The (AVM) pulls harmful chemical vapors away from the user and passes it through this carbon filter, which neutralizes the vapors. This filter needs to be changed every 6 months.

HIGH-LEVEL DISINFECTANT

Rapicide (4 - one gallon bottles/case) - ML02-0059

The DSD uses 4 gallons in each reservoir; it is changed every 28 days or sooner if it fails the MEC potency test. Always make sure you have an extra gallon for adding (i.e. topping off) as there will be some loss of disinfectant from normal usage.

Rapicide Glutaraldehyde Indicator Test Strip (2 bottles/case, each bottle has 60 strips) - ML02-0120

Usage depends on how many procedures/day.

DETERGENT AND WIPES

Intercept Detergent (4 - one gallon bottles/case) - ML02-0106

Endoscope Detergent for pre-cleaning, proven to reduce biofilm. Use only one ounce (1 full pump) of Intercept Detergent in three (3) gallons of water.

Intercept Wipes (4 containers/case, each bottle has 60 strips) - ML02-0107

Intercept detergent towelettes are a low cost way to wipe down contaminated endoscopes and reduce biofilm.

MISCELLANEOUS

Printer Paper (one each) - 1-4-501

Single ply with spindle hole. Usage is depending on the volume of procedures.

Printer Ribbon (one each) - 1-4-503

Black/red ink printer ribbon. Usage is depending on the volume of procedures.

Accessory Bags (set of 2) - CB12-0002

Accessory bag for placement of endoscope accessories, such as valves.

Barcode Tags (one set of #100-199) - 67199-157

Adhesive barcode tags include duplicates of each number.

Barcode Tags (one set of #200-299) - 67199-158

Adhesive barcode tags include duplicates of each number.

Silicone Oil (one each) - 17599-496

Used for lubricating connector O-rings. Lubricate O-rings at least once a week.

To order, call MEDIVATORS directly at: 1-800-444-4729, option 1 • www.minntech.com/medivators

Endoscope reprocessing the way it should be

MEDIVATORS