



COMPETENCY ASSESSMENT CER-1 & CER-2

Employee will demonstrate competency in using the Medivators CER-1 and CER-2 Endoscope Reprocessor.

EMPLOYEE NAME: _____

	CRITICAL SKILLS DEMONSTRATION	DATE REVIEWED	MET	NOT MET	COMMENTS
1.	Uses proper Personal Protective Equipment (PPE).				
2.	Verbalizes start of day process and turns water supply ON.				
3.	Performs Daily QA water flow verification and documents results.				
4.	Performs Daily QA air flow verification and documents results.				
5.	Checks and cleans drain screen.				
6.	Verifies and attains proper disinfectant level in reservoirs.				
7.	Verifies correct disinfectant temperature in reservoir (if applicable); document temperature.				
8.	Confirms disinfectant is within its reuse period.				
9.	Tests the disinfectant for Minimum Effective Concentration (MEC) prior to each use/cycle.				
10.	Properly places endoscope(s) in CER basin.				
11.	Selects correct hookup for each endoscope model.				
12.	Verifies hookup has no holes/tears/cracks or has not been modified prior to use.				
13.	Places accessories in mesh bag and in CER basin.				
14.	Successfully runs FULL, HLD/RINSE, and MANUAL cycles.				
15.	Verifies cycle completed successfully before removing endoscope.				
16.	Verbalizes the function of the control panel buttons.				
17.	Successfully troubleshoots alarm conditions; verbalizes manual button use to transfer basin fluid if CER stops mid-cycle.				
18.	Successfully dumps and reloads disinfectant from/to reservoir.				
19.	Changes 1 micron water pre-filter and 0.2 micron filter; documents changes.				
20.	Successfully sanitizes water lines after 0.2 micron water filter change; documents sanitization.				
21.	Verbalizes where to locate Medivators phone number for ordering supplies, technical support and servicing.				
22.	Utilizes Quick Start Guide, Daily QA Checklist, Fill & Empty Guide, Filter Log, Hook-up Guide, Reorder Guide, and User manual.				
23.	Verbalizes end of day shutdown; Shut water supply OFF, clean CER system using lint-free cloth.				

PASS: REDO:

EMPLOYEE SIGNATURE: _____ DATE: _____

REVIEWER'S SIGNATURE: _____ DATE: _____

www.minntech.com/medivators • Toll Free: 1-800-444-4729

Endoscope reprocessing the way it should be

MEDIVATORS