



TO EMPTY DISINFECTANT FROM RESERVOIR

- Turn **POWER ON** to the CER unit.
- For heated disinfectant (i.e. Rapicide), turn **POWER OFF** to the reservoir heater.
- Close and lock CER basin lid.
- Press **MANUAL, LCG/IN, START**. It will take about 5 minutes to transfer the disinfectant from the reservoir to the basin. The transfer will automatically stop when the basin is full. Some disinfectant may remain in the reservoir.
- Press **MANUAL, DRAIN, START**. Disinfectant will now drain from the basin out to the drain.
- Press **STOP** when the basin is empty.
- Repeat process by pressing **MANUAL, LCG/IN, START** to transfer the remaining disinfectant from the reservoir to the basin. You may need to lift one end of the reservoir to completely empty it.
- When the reservoir is empty, you will hear a sucking sound, then press **STOP**.
- Press **MANUAL, DRAIN, START**. Disinfectant will again drain from basin out to the drain.
- Press **STOP** when basin is empty.
- Put on protective gloves.
- Remove access lid from reservoir.
- Wipe out inside of reservoir with absorbent lint-free cloths/towels moistened with tap water.

www.minntech.com/medivators • Toll Free: 1-800-444-4729

3 METHODS OF FILLING DISINFECTANT RESERVOIR

1

LOADING Disinfectant using Transfer Pump

- Plug the Transfer Pump into a power outlet.
- Wear Personal Protective Equipment (PPE).
- Place the rigid end of the pump tubing into the new HLD container, and connect the opposite tubing end to the inlet of the Transfer Pump.
- Connect the other pump tubing between the outlet of the pump and the reservoir.
- Turn transfer pump **ON** at the switch.
- Fill the reservoir until the new HLD has been fully transferred.
 - Model CER-1, 4 gallons of HLD is required.
 - Model CER-2, 5 gallons of HLD is required.
- Turn the pump **OFF** and disconnect tubing.
- Flush the Transfer Pump with clear water.
- For heated disinfectant (i.e. Rapicide), turn **POWER ON** to the reservoir heater.

2

LOADING Disinfectant from CER Basin

- Wear Personal Protective Equipment (PPE).
- Turn **POWER ON** to the CER unit.
- Press **MANUAL, LCG/OUT, START**.
- For a CER-1, pour 4 gallons of HLD into the basin.
For a CER-2, pour 5 gallons of HLD into the basin.
- Once all of the disinfectant is in the reservoir, press **STOP**.
- To rinse the basin of disinfectant residual, press **FULL**, then **START**.
- Once the wash cycle is finished, press **STOP**.
- For heated disinfectant (i.e. Rapicide), turn **POWER ON** to the reservoir heater.

3

LOADING Disinfectant Directly into Reservoir

- Wear Personal Protective Equipment (PPE).
- For a CER-1, pour 4 gallons of HLD into the reservoir.
For a CER-2, pour 5 gallons of HLD into the reservoir.
- For heated disinfectant (i.e. Rapicide), turn **POWER ON** to the reservoir heater.