



FILTER PACKAGES

CER Filter Package - MF01-0046

Save money and time by purchasing a filter package.

Filter package includes: 1 Micron Water Filter - 4 each (MF01-0014), 0.2 Micron Absolute Bacterial Water Filter - 2 each (CF01-0018)

INDIVIDUAL FILTERS

0.2 Micron Absolute Bacteria Water Filter (one each) - CF01-0018

This filter is located between the incoming water line and your CER machine. When looking at the water filtration system, it is the filter on the left. It needs to be changed every 6 months or sooner if poor water conditions exist. Always keep one on hand and a water line disinfectant process must be performed when changing this filter.

1 Micron Water Filter (one each) - MF01-0014

This nominal filter is located between the incoming water line and your CER machine. When looking at the water filtration system, it is the filter on the right. Change is pressure related, water pressure should be 40 psi or above. Always keep one on hand. Change at least every 3 months even if pressure remains above 40 psi.

Charcoal Replacement Filter for Active Vapor Management System (one each) - MF01-0055

This filter is contained within the Active Vapor Management System (AVMS) and draws disinfectant vapors away from the user. This patented charcoal filter needs to be changed every 6 months.

HIGH-LEVEL DISINFECTANT

Rapicide (4 - one gallon bottles/case) - ML02-0059

The CER-1 uses 4 gallons; CER-2 uses 5 gallons. It is changed every 28 days or sooner if it fails the MEC potency test. Always make sure you have an extra gallon for adding (i.e. topping off) as there will be some loss of disinfectant from normal usage.

Rapicide Glutaraldehyde Indicator Test Strips (2 bottles/case, each bottle has 60 strips) - ML02-0120

Usage depends on how many procedures/day.

DETERGENT AND WIPES

Intercept Detergent (4 - one gallon bottles/case) - ML02-0106

Endoscope Detergent for pre-cleaning, proven to reduce biofilm. Use only one ounce (1 full pump) of Intercept Detergent in three (3) gallons of water.

Intercept Wipes (4 containers/case, each container has 100 pre-saturated wipes) - ML02-0107

Intercept detergent towelettes are a low cost way to wipe down contaminated endoscopes and reduce biofilm.

MISCELLANEOUS

Accessory Bags (set of 2) - CB12-0002

Accessory bag for placement of endoscope accessories, such as valves.

Silicone Oil (one each) - 17599-496

Used for lubricating connector O-rings. Lubricate O-rings at least once a week.

To order, call MEDIVATORS directly at: 1-800-444-4729, option 1 • www.minntech.com/medivators

Endoscope reprocessing the way it should be

MEDIVATORS