



COMPETENCY ASSESSMENT DSD-201

Employee will demonstrate competency in using the Medivators DSD Endoscope Reprocessor.

EMPLOYEE NAME: _____

	CRITICAL SKILLS DEMONSTRATION	DATE REVIEWED	MET	NOT MET	COMMENTS
1.	Uses proper Personal Protective Equipment (PPE).				
2.	Verification and attaining proper level of disinfectant in reservoir.				
3.	Verification of correct temperature of disinfectant, if applicable.				
4.	Confirms disinfectant is within its reuse period				
5.	Tests the disinfectant for Minimum Effective Concentration (MEC) prior to each use/cycle.				
6.	Properly places endoscopes in DSD basin.				
7.	Selects correct hookup for each scope model.				
8.	Verifies hookup has no holes/tears/cracks or has not been modified prior to use.				
9.	Places accessories in mesh bags and in DSD basin.				
10.	Correctly selects the station and enters ID (identification) data for scope #, operator, patient and physician.				
11.	Successfully starts a cycle.				
12.	Correctly presses ENTER, if MEC passes or CANCEL, if MEC fails, when DSD propts "LCG test pass?".				
13.	Successfully interrupts and restarts a cycle.				
14.	Verifies the cycle has completed successfully before removing the scope.				
15.	Verbalizes the function of the control panel buttons.				
16.	Identifies the cycle has completed successfully before removing the scope.				
17.	Uses SETUP button to access system functions.				
18.	Correctly changes printer paper and ribbon.				
19.	Successfully dumps and reloads the disinfectant from/to the reservoir.				
20.	Changes disinfectant and air filters at appropriate intervals; documents changes.				
21.	Changes the water pre filters (1µ and .45µ) and internal water filter (.2µ) change; documents sanitization.				
22.	Successfully sanitizes water lines after each internal water filter (.2µ) change; documents sanitization.				
23.	Verbalizes where to locate Medivators phone number for technical support, ordering DSD supplies and services.				
24.	Utilizes Quick Start Guide, Daily Checklist, Dump & Load Guide, Filter Log, Hook-up Guide, User Manual, as appropriate.				

PASS: REDO:

EMPLOYEE SIGNATURE: _____ DATE: _____

REVIEWER'S SIGNATURE: _____ DATE: _____

www.minntech.com/medivators • Toll Free: 1-800-444-4729

Endoscope reprocessing the way it should be

MEDIVATORS