



“PREFERRED AND SAFEST METHOD”

AUTOMATICALLY LOADING DISINFECTANT BY USING THE DSD’S INTERNAL TRANSFER PUMP

Refer to Product Bulletin 50096-348 for more detailed instructions.

1

LOCATE the transfer pump tubing segment with the red male end and the female cream-colored end (figure 1).

2

CONNECT the red male end of the transfer pump tubing to the red female connector located at the top of the reservoir which needs to be loaded with disinfectant.

3

CONNECT the cream-colored female end of the transfer pump tubing to the cream-colored male end of the 3-way valve located on the transfer pump.

4

LOCATE the transfer pump tubing segment with the red female end and the long rigid end with rubber seal (figure 1).

5

CONNECT the red female end of the transfer pump tubing to the red male end of the disinfectant filter on the “B” side.

6

PLACE the rigid end of the transfer pump tubing with rubber seal into a disinfectant container (figure 2).

7

ROTATE the 3-way valve handle of the transfer pump until it points out from sidewall of DSD cabinet which also indicates the direction that fluid will flow (figure 2).

8

LOCATE the manual pump switch on the right side of the cabinet wall labeled Chemical Loading Switch.

9

PRESS and hold switch until all the disinfectant has been pumped into the reservoir. Repeat with new gallons of disinfectant until four (4) gallons have been pumped into the reservoirs.

10

ROTATE the 3-way valve handle back to its original position.

11

RETURN all DSD tubing connections back to their original position.

12

RINSE the Transfer Pump Tubing with clean water; hang to dry.

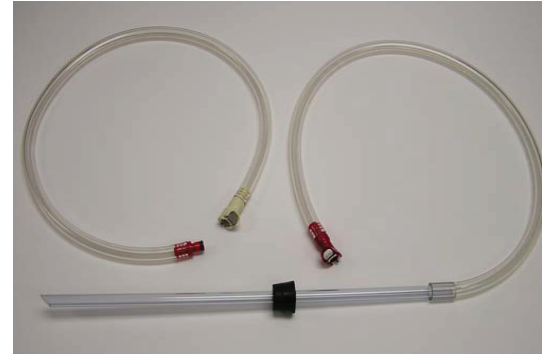


Figure 1. Transfer Pump Tubing Kit

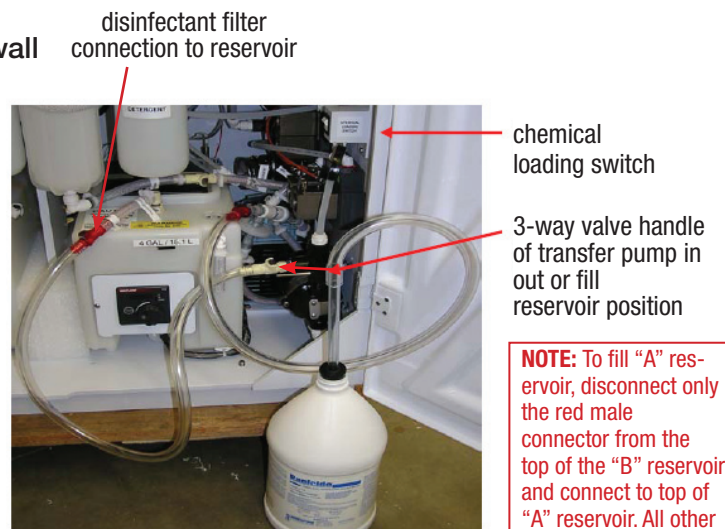


Figure 2. Loading disinfectant into DSD reservoir

NOTE: To fill “A” reservoir, disconnect only the red male connector from the top of the “B” reservoir and connect to top of “A” reservoir. All other connectors remain the same.



AUTOMATIC DISINFECTANT DUMP

- 1** **PRESS** “STATION SELECT A” or “B”
- 2** **PRESS** “DISINFECTANT DUMP”
- 3** **PRESS** “START”
- 4** DSD prompts: “ATTACH RESTRICTOR”
(short connector #ASMI - 0137)
- 5** **ATTACH** Restrictor; **PRESS** “START”
- 6** After dumping disinfectant, a basin rinse will be performed.
- 7** **REPLACE** disinfectant filter
- 8** **WIPE** reservoir with a damp, lint-free cloth
- 9** **REMINDER:** Pull air tank ring
(inside cabinet in left rear corner)

LOADING DISINFECTANT FROM THE TOP BASIN

- 1** **PRESS** “STATION SELECT A” or “B”
- 2** **PRESS** “SET UP”
- 3** **PRESS** “1”
- 4** **PRESS** “ENTER”
- 5** **PRESS** “START”
- 6** DSD prompts: “ATTACH RESTRICTOR”
(short connector #ASMI - 0137)
- 7** **ATTACH** Restrictor; **PRESS** “START”
- 8** **LOAD** disinfectant (4) gallons, by pouring in the top basin.
- 9** When all disinfectant is **LOADED** and in the reservoir, **CLOSE** lid, press **CANCEL**, then **ENTER**.

OTHER USEFUL FUNCTIONS

- To check temperature: **PRESS** “SET UP”, “13”, “ENTER”.
- To see remaining time: **PRESS** “SET UP”, “17”, “ENTER”.
- To abort a cycle: **PRESS** “CANCEL”, then “ENTER”.

Medivators Website “Resource Center”

Go to: www.minntech.com/medivators, Select “Resource Center” and “User Library” for detailed user guides and hook-up matrices, report forms and logs, and product bulletins

Medivators Customer and Technical Support Toll Free: Phone: 1-800-444-4729 • FAX: 1-866-421-7696