



# COMPETENCY ASSESSMENT DSD EDGE

Employee will demonstrate competency in using the Medivators DSD EDGE Endoscope Reprocessor.

EMPLOYEE NAME: \_\_\_\_\_

	CRITICAL SKILLS DEMONSTRATION	DATE REVIEWED	MET	NOT MET	COMMENTS
1.	Use proper Personal Protective Equipment (PPE)				
2.	Confirms Rapicide PA is within its open container use period (21 days)				
3.	Confirms the water is at the proper temperature of 35°C by using SETUP 13 function				
4.	Successfully explains when to use the SETUP 41 Incoming Water Temperature utility and how to execute.				
5.	Proper placement of endoscope(s) in basin.				
6.	Selects correct hookup for each scope model.				
7.	Verification that hookup has no holes /tears/cracks/ modifications prior to use				
8.	Place accessories in mesh bag and properly position bag in DSD EDGE basin.				
9.	Correctly selects the station and enters ID (identification) data for scope #, operator, patient and physician.				
10.	Successfully starts a cycle.				
11.	Successfully interrupts and restarts a cycle.				
12.	When prompted, tests the Rapicide PA with the Rapicide PA test strip in the sample port and reads results.				
13.	Presses HLD PASS button when the test strip confirms the Rapicide PA was effective, or HLD FAIL button if not				
14.	Verifies the cycle has completed successfully before removing the scope.				
15.	Verbalizes the function of the control panel buttons.				
16.	Identifies and successfully troubleshoots error messages.				
17.	Uses SETUP button to access system functions.				
18.	Correctly changes printer paper and ribbon.				
19.	Successfully replaces the Rapicide PA Parts A and B disinfectant containers.				
20.	Checks and cleans the reusable disinfectant filter monthly; documents on filter log.				
21.	Changes air filters every three months; documents changes.				
22.	Changes the water pre-filters (1µ and .45µ) and internal water filter (.2µ); documents changes.				
23.	Successfully disinfects water lines weekly and after each internal water filter (.2µ) change; documents disinfection.				
24.	Verbalizes where to locate Medivators phone number for technical support, ordering DSD supplies and service.				
25.	Utilizes Quick Start Guide, Daily Checklist, Filter Log, Hook-up Guide and User Manual, as appropriate.				

PASS:  REDO:

EMPLOYEE SIGNATURE: \_\_\_\_\_ DATE: \_\_\_\_\_

REVIEWER'S SIGNATURE: \_\_\_\_\_ DATE: \_\_\_\_\_

www.minntech.com/medivators • Toll Free: 1-800-444-4729

Endoscope reprocessing the way it should be

# MEDIVATORS