



The following table lists the various set up functions that can be run on the DSD EDGE. The table shows the entry number and the corresponding function.

## Press: Set Up, Entry #, ENTER

ENTRY NUMBER	FUNCTION
2	Set Date
3	Set Time
4	Display software version number
5	Input a custom program
6	Run waterline disinfection program
8	Display log information
9	Disable logging
10	Clear the log
13	Display temperatures
16	Display the disinfectant cycle count
17	Display the cycle time remaining
18	Display the state time
21	Print the entire log
22	Cancel print run
25	Print the last disinfection cycle
28	Set delayed start time
29	Enable delayed start
33	Enable automatic log printout
41	Run incoming water temperature utility
42	Display state and cycle time remaining
43	Open main water valve (for filter purging)