

Rapicide[®] PA

REF
ML02-0126

HIGH LEVEL DISINFECTANT

USE ONLY WITH AN AUTOMATED ENDOSCOPE REPROCESSING SYSTEM

KEEP OUT OF REACH OF CHILDREN
DANGER



READ DIRECTIONS FOR USE AND MSDS FOR THE RAPICIDE PA AND THE AUTOMATED ENDOSCOPE REPROCESSING SYSTEM

DIRECTIONS FOR USE

WARNING

For endoscope reprocessing in a Validated Automated Reprocessing System only.

Part A and B can not be used separately.

PACKAGING:

4 Bottles per Case (20 Liters).

2 Bottles of Part A X 5 Liters

2 Bottles of Part B X 5 Liters

Catalog #: ML02-0126

ARTG # 175066

COMPOSITION:

Activated solution (A+B): active ingredients:
trisodium phosphate .091 g; peracetic acid .105 g;
hydrogen peroxide .420 g; inert ingredients and water to 100 mL.

DESCRIPTION OF ACTIVATED SOLUTION

Part A & Part B -

Concentrated high level disinfecting solution, with sporicidal, tuberculocidal, virucidal, fungicidal, and bactericidal activity when used in a validated Automated Endoscope Reprocessing System.

Rapicide[®] PA

REF
ML02-0126

HIGH LEVEL DISINFECTANT

USE ONLY WITH AN AUTOMATED ENDOSCOPE REPROCESSING SYSTEM

KEEP OUT OF REACH OF CHILDREN
DANGER



READ DIRECTIONS FOR USE AND MSDS FOR THE RAPICIDE PA AND THE AUTOMATED ENDOSCOPE REPROCESSING SYSTEM

DIRECTIONS FOR USE

WARNING

For endoscope reprocessing in a Validated Automated Reprocessing System only.

Part A and B can not be used separately.

PACKAGING:

4 Bottles per Case (20 Liters).

2 Bottles of Part A X 5 Liters

2 Bottles of Part B X 5 Liters

Catalog #: ML02-0126

ARTG # 175066

COMPOSITION:

Activated solution (A+B): active ingredients:
trisodium phosphate .091 g; peracetic acid .105 g;
hydrogen peroxide .420 g; inert ingredients and water to 100 mL.

DESCRIPTION OF ACTIVATED SOLUTION

Part A & Part B -

Concentrated high level disinfecting solution, with sporicidal, tuberculocidal, virucidal, fungicidal, and bactericidal activity when used in a validated Automated Endoscope Reprocessing System.

Endoscope reprocessing the way it should be

MEDIVATORS

Manufactured in the USA by:



Minntech Corporation
14605 28th Avenue North
Minneapolis, MN 55447 U.S.A.
Toll Free: (800) 328-3340
email: info@minntech.com



Minntech BV
Sourethweg 11
6422PC Heerlen
The Netherlands
Tel: +31 45 5471 471
Fax: +31 45 5429 695
email: info@minntechbv.com

ECIREP

50097-400/A

Minntech Asia / Pacific Pte Ltd
1A International Business Park
#05-01 Singapore 609933
Tel: +65 6227 9698
Fax: +65 6225 6848
email: info@minntech.com.sg

Endoscope reprocessing the way it should be

MEDIVATORS

Manufactured in the USA by:



Minntech Corporation
14605 28th Avenue North
Minneapolis, MN 55447 U.S.A.
Toll Free: (800) 328-3340
email: info@minntech.com



Minntech BV
Sourethweg 11
6422PC Heerlen
The Netherlands
Tel: +31 45 5471 471
Fax: +31 45 5429 695
email: info@minntechbv.com

ECIREP

50097-400/A

Minntech Asia / Pacific Pte Ltd
1A International Business Park
#05-01 Singapore 609933
Tel: +65 6227 9698
Fax: +65 6225 6848
email: info@minntech.com.sg

PRODUCT EFFICACY:

For high level disinfection of endoscopes. Contact time 5 minutes at 30 °C.

DIRECTIONS FOR USE:

It is a violation of Federal (USA) law to use this product in a manner inconsistent with its labeling.

Connect one bottle of Part A and one bottle of Part B to an automated endoscope reprocessor per user manual instructions. Ensure that the automated endoscope reprocessor mixes the correct volumes of water, Part A and Part B to achieve at least 850 ppm of peracetic acid (Minimum Recommended Concentration). Check the use solution with a Rapicide PA Test Strip to ensure that it is above MRC. WARNING - Do not mix with other detergents or chemicals.

COMPATIBILITY:

The activated solution has been shown to be highly compatible with the materials of endoscopes and certain Automated Endoscope Reprocessing Systems. Data is on file with the manufacturer.

STORAGE: Keep away from children. Keep in a dry place and store at room temperature away from heating sources. Expiration date applies to properly preserved unused product. Do not use after expiration date. The product may be used for twenty-one (21) days after the containers are opened. DO NOT use after that time has expired.

PART A:

R: 8-24/21/22-35-41

S: (1/2)-3/7-17-26-28-36/37/39-45-61

PART B:

R: 34

S: 36/37/39

The activated and diluted (A+B) product does not require additional specific safety measures for people or the environment. Dispose of any unused material to include the container in accordance with all applicable European and Local regulations. For 24-hour chemical emergency assistance, call CHEMTREC at 1(703) 527-3887. See Material Safety Data Sheet.

PRODUCT EFFICACY:

For high level disinfection of endoscopes. Contact time 5 minutes at 30 °C.

DIRECTIONS FOR USE:

It is a violation of Federal (USA) law to use this product in a manner inconsistent with its labeling.

Connect one bottle of Part A and one bottle of Part B to an automated endoscope reprocessor per user manual instructions. Ensure that the automated endoscope reprocessor mixes the correct volumes of water, Part A and Part B to achieve at least 850 ppm of peracetic acid (Minimum Recommended Concentration). Check the use solution with a Rapicide PA Test Strip to ensure that it is above MRC. WARNING - Do not mix with other detergents or chemicals.

COMPATIBILITY:

The activated solution has been shown to be highly compatible with the materials of endoscopes and certain Automated Endoscope Reprocessing Systems. Data is on file with the manufacturer.

STORAGE: Keep away from children. Keep in a dry place and store at room temperature away from heating sources. Expiration date applies to properly preserved unused product. Do not use after expiration date. The product may be used for twenty-one (21) days after the containers are opened. DO NOT use after that time has expired.

PART A:

R: 8-24/21/22-35-41

S: (1/2)-3/7-17-26-28-36/37/39-45-61

PART B:

R: 34

S: 36/37/39

The activated and diluted (A+B) product does not require additional specific safety measures for people or the environment. Dispose of any unused material to include the container in accordance with all applicable European and Local regulations. For 24-hour chemical emergency assistance, call CHEMTREC at 1(703) 527-3887. See Material Safety Data Sheet.