# Instruction for Use

MEDIVATORS®  
**Rapicide**®  
High-Level Disinfectant & Sterilant

## EN INDICATIONS FOR USE

### 1. Germicidal Level of Activity

RAPICIDE High-Level Disinfectant and Sterilant is a liquid chemical sterilant and a high-level disinfectant when used according to Directions for Use. STERILANT - RAPICIDE High-Level Disinfectant and Sterilant is a sterilant when used or reused, in a legally marketed Automated Endoscope Reprocessor according to Directions for Use, at a minimum recommended concentration (MRC) of 1.5% glutaraldehyde at 35°C (95°F), not to exceed 28 days, with a minimum contact or immersion time of at least 7 hours and 40 minutes.

HIGH-LEVEL DISINFECTANT - RAPICIDE High-Level Disinfectant and Sterilant is a high-level disinfectant when used or reused, in a legally marketed Automated Endoscope Reprocessor according to Directions for Use, at a minimum recommended concentration (MRC) of 1.5% glutaraldehyde at 35°C (95°F), not to exceed 28 days, with a minimum contact or immersion time of at least 5 minutes.

### 2. Reuse Period

RAPICIDE High-Level Disinfectant and Sterilant, at a minimum recommended concentration (MRC) of 1.5% glutaraldehyde, has demonstrated efficacy in the presence of organic matter and microbiological burden during reuse. The solution can be reused as long as the minimum recommended concentration (MRC) of glutaraldehyde is 1.5% or greater. Maintain the solution at 35°C (95°F) over the 28-day reuse period. In no case should the solution be reused longer than 28 days. DO NOT RELY SOLELY ON DAYS IN USE. The concentration of this product during its use-life must be verified by utilizing Medivators® Rapicide Glutaraldehyde Indicator Test Strips, (order number ML02-0120), chemical indicator strips to determine that at least the minimum recommended concentration (MRC) of 1.5% glutaraldehyde is present.

### 3. Material Compatibility

RAPICIDE High-Level Disinfectant and Sterilant is compatible with the following reusable devices and materials: flexible endoscopes, rubber, plastic, many types of metals, such as stainless steel, carbon steel, aluminum, and plated metals such as chrome plating.

RAPICIDE High-Level Disinfectant and Sterilant is compatible with detergents which are mild in pH, low foaming, and easily rinsed from equipment. Detergents that are either highly acid or alkaline are contraindicated as pre-cleaning agents since improper rinsing could affect the efficacy of RAPICIDE High-Level Disinfectant and Sterilant by altering its pH.

## CONTRAINDICATIONS

### 1. Sterilant Usage

Routine biological monitoring is not feasible with RAPICIDE High-Level Disinfectant and Sterilant, and therefore RAPICIDE High-Level Disinfectant and Sterilant should NOT be used to sterilize reusable medical devices that are compatible with other available methods of sterilization that can be biologically monitored (e.g., heat, ethylene oxide, or gas plasma).

### 2. High-Level Disinfectant Usage

RAPICIDE High-Level Disinfectant and Sterilant should NOT be used to high-level disinfect a semi-critical device when sterilization is practical.

## WARNINGS

RAPICIDE High-Level Disinfectant and Sterilant is hazardous to humans and domestic animals.

Misuse of product NOT in accordance with the Directions for Use may result in chemical exposure.

### DANGER: Keep Out of Reach of Children.

**WARNING:** Rapicide can ONLY be used at ambient (room) temperature for specific sanitization processes. It is **NOT** to be used at ambient (room) temperature for high-level disinfection of endoscopes.

1. Direct contact is corrosive to exposed tissue, causing eye damage and skin irritation/damage. Do not get into eyes, on skin, or on clothing.
2. Avoid contamination of food.
3. Use in well-ventilated areas in an Automated Endoscope Reprocessor with an effective vapor containment system to manage vapors. In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, get medical attention. Harmful if swallowed. Drink large quantities of water and call a physician immediately. Probable mucosal damage from oral exposure may contraindicate the use of gastric lavage. Emergency, safety, or technical information about RAPICIDE High-Level Disinfectant and Sterilant can be obtained by calling Medivators 1-800-328-3340 or 1-763-553-3300.

## PRECAUTIONS

1. Nitrile gloves, eye protection, face masks, and liquid-proof gowns should be worn when cleaning and sterilizing/disinfecting soiled devices.
2. The user is cautioned to minimize exposure to vapors at elevated temperature by using RAPICIDE High-Level Disinfectant and Sterilant in an Automated Endoscope Reprocessor with an effective vapor containment system and placed in a well-ventilated area.
3. Contaminated reusable devices **MUST BE THOROUGHLY CLEANED** prior to disinfection or sterilization, since residual contamination will decrease effectiveness of the germicide.
4. The user **MUST** adhere to the Direction for Use since any modification will affect the safety and effectiveness of the germicide.
5. The reusable device manufacturer should provide the user with a validated reprocessing procedure for that device using RAPICIDE High-Level Disinfectant and Sterilant.
6. The use of RAPICIDE High-Level Disinfectant and Sterilant in Automated Endoscope Reprocessors must be part of a validated reprocessing procedure supplied by the Automated Endoscope Reprocessor manufacturer to ensure that a minimum contact or immersion time of 5 minutes at 35°C (95°F) is provided for high-level disinfection, and 7 hours and 40 minutes at 35°C (95°F) for sterilization.
7. Use Medivators® Rapicide Glutaraldehyde Indicator Test Strips, (order number ML02-0120), chemical indicator strips to monitor glutaraldehyde concentration prior to each reprocessing cycle in order to detect unexpected dilution.

## DIRECTION FOR USE

- A. **Cleaning and Decontamination:** Blood, other body fluids and lubricants must be thoroughly cleaned from the surfaces and lumens of all devices before reprocessing in the Rapicide disinfectant. Refer to the manufacturer's labeling for instructions on disassembly, decontamination, cleaning and leak testing of their equipment. Before immersion in Rapicide, thoroughly clean, rinse and rough dry devices before reprocessing in Rapicide High-Level Disinfectant and Sterilant.
- B. **Preparation and Usage:** Rapicide High-Level Disinfectant and Sterilant is supplied ready-to-use and does not require activation. Record the date the container was opened on the container label or in a logbook. After opening, the solution may be stored in the original container for up to 28 days (providing the 28 days does not extend past the expiration date on the container) until used.

Check the concentration of the Rapicide glutaraldehyde solution prior to each reprocessing cycle using Medivators Rapicide Glutaraldehyde Indicator Test Strips and discard the solution when less than 1.5% glutaraldehyde is present.

- C. **High-Level Disinfection and Sterilization Procedure:** It is a violation of International Law to use this product in a manner inconsistent with its labeling. Monitor the minimum recommended concentration (MRC) of glutaraldehyde with Medivators Rapicide Glutaraldehyde Indicator Test Strips prior to each reprocessing cycle.

Sterilization

- a. Wearing proper personal protective equipment, place pre-cleaned devices into an Automated Endoscope Reprocessor (AER) in accordance with the manufacturer's instructions. Select a validated sterilization cycle that provides for a minimum sterilant contact or immersion time of 7 hours and 40 minutes at 35°C to eliminate all microorganisms. Select an automated rinsing cycle that provides thorough rinsing equivalent to the device manufacturer's rinsing instructions and consistent with the automated processing rinsing instructions provided below.

High-Level Disinfection

- b. Wearing proper personal protective equipment, please pre-cleaned devices into an Automated Endoscope Reprocessor (AER) in accordance with the manufacturer's instructions. Select a validated high-level disinfection cycle that provides for a minimum disinfectant contact or immersion time of 5 minutes at 35°C. Select an automated rinsing cycle that provides thorough rinsing equivalent to the device manufacturer's rinsing instructions and consistent with the automated processing rinsing instructions provided below.

- c. Select a validated automated rinse cycle in accordance with the Automated Endoscope Reprocessor manufacturer's instructions. Ensure that the automated rinse cycle selected will thoroughly rinse the device including all channels with large volumes of rinse water equivalent to the device manufacturer's recommendations. Each rinse should be a minimum of 1 minute in duration unless otherwise noted by the device manufacturer. Ensure that a fresh supply of water is used for each of the 2 required rinses with Rapicide. Do not reuse the water for rinsing or any other purpose.

- D. **Sanitization:** Rapicide can be used at ambient (room) temperature in order to perform sanitization processes including decontamination of AER waterlines and Scope Buddy Endoscope Flushing Aids. In these specific applications, Rapicide must be used at a minimum recommended concentration (MRC) of 1.5% glutaraldehyde and a minimum contact time of 1 hour. Follow the AER or Scope Buddy directions for performing the sanitization process.

**WARNING:** Rapicide can **ONLY** be used at ambient (room) temperature for specific sanitization processes. It is **NOT** to be used at ambient (room) temperature for high-level disinfection or sterilization of any other medical devices.

Disinfected and/or sterilized reusable devices are either to be immediately used or stored in a manner to minimize recontamination. Note that only terminal sterilization (sterilization in a suitable wrap) provides maximum assurance against recontamination. Refer to the reusable device/equipment manufacturer's labeling for additional storage and/or handling instructions.

**STORAGE CONDITIONS AND EXPIRATION DATE**

1. Ready-to use RAPICIDE High-Level Disinfectant and Sterilant should be stored in its original sealed container at a controlled room temperature of 15°C to 30°C (59°F to 86°F). Once opened, the contents of the container can be used for up to 28 days, as long as the glutaraldehyde concentration is greater than the MRC of 1.5%.
2. The expiration date of RAPICIDE High-Level Disinfectant and Sterilant will be found at the bottom of the bottle label. Do not utilize beyond the expiration date.
3. The use period of RAPICIDE High-Level Disinfectant and Sterilant should never exceed 28 days. Medivators Rapicide Glutaraldehyde Indicator Test Strips, (order number ML02-0120), chemical indicator strips will indicate at any time if the concentration of glutaraldehyde is below the MRC of 1.5%. **DO NOT RELY SOLELY ON DAYS OF USE.**

**EMERGENCY AND TECHNICAL PRODUCT INFORMATION**

Emergency, safety, or technical information about RAPICIDE High-Level Disinfectant and Sterilant can be obtained by calling Medivators at 1-800-328-3340 or 1-763-553-3300.

**USER PROFICIENCY**

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling. The user should be adequately trained in the decontamination and disinfection or sterilization of medical devices and the handling of toxic substances such as liquid chemical germicides. Additional information about RAPICIDE High-Level Disinfectant and Sterilant can be obtained from Medivators at 1-800-328-3340 or 1-763-553-3300.

**DISPOSAL INFORMATION**

Germicide Disposal Discard residual solution in drain or according to local, state, and European regulations.

Container Disposal Do not reuse empty container. Rinse thoroughly with water and dispose in trash.

**COMPONENT INFORMATION**

Trade Name and Synonyms: RAPICIDE High-Level Disinfectant and Sterilant  
Chemical Family: Di-aldehydes  
Chemical Name and Synonyms: Glutaraldehyde or 1,5 Pentanedial-2%  
CAS No.: 111-30-8

Hazardous Materials  
Identification System (NIOSH)  
2 Health  
0 Flammability  
0 Reactivity  
B Personal Protection

**HOW SUPPLIED**

Order Information: RAPICIDE High-Level Disinfectant and Sterilant  
4 X 1 gallon bottles per case  
Product Code: ML02-0059

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© 2013 MEDIVATORS Inc. 50097-852/D
MEDIVATORS Inc. Headquarters
MEDIVATORS Asia/Pacific Pte Ltd