

MEDIVATORS®

# RAPICIDE® PA

High-Level Disinfectant & Sterilant



**PART B Rapicide PA** (RTU) ML02-0125  
**READY-TO-USE**  
HIGH LEVEL DISINFECTANT & STERILANT  
FOR ENDOSCOPE REPROCESSING & AUTOMATED REPROCESSING SYSTEMS FOR MANUAL DISINFECTION OF MEDICAL DEVICES

**Ingredients:**  
Sodium peroxide 1.02%  
Hydrogen Peroxide 11.02%  
Peracetic Acid 18.98%  
Water 100.0%

**NET CONTENTS:**  
4.775 Liters  
(1.26 Gallons)

**71175**  
**11/05**  
**2/05**

MEDIVATORS®  
www.minitech.com/medivators

**PART A Rapicide PA**  
HIGH LEVEL DISINFECTANT  
FOR ENDOSCOPE REPROCESSING IN AN AUTOMATED REPROCESSING SYSTEM  
FOR MANUAL DISINFECTION OF MEDICAL DEVICES

**KEEP OUT OF REACH OF CHILDREN**

Caution: Causes irritable eye discharge and skin irritation. Also harmful if swallowed. Avoid contact with eyes, nose, mouth, and skin. Wash thoroughly with soap and water after use. Use eye goggles, face shield or safety glasses and protective gloves when handling. Rinse equipment thoroughly with water after use.

**PART A Rapicide PA** Catalog No. ML02-0117  
**HIGH LEVEL DISINFECTANT**  
USE ONLY WITH ADVANTAGE™ PLUS OR DSD EDGE™ ENDOSCOPE REPROCESSING SYSTEMS

**KEEP OUT OF REACH OF CHILDREN DANGER**

**NET CONTENTS:**  
5.0 Liters  
(1.32 Gallons)

**Active Ingredients:**  
Hydrogen Peroxide 22.0%  
Peracetic Acid 5.0%  
Inert Ingredients 73.0%  
Total 100.0%

MEDIVATORS®  
Manufactured in the USA by  
Minitech Corporation  
10000 W. 15th St., Suite 100  
Denver, CO 80202  
Tel: 303.440.1000  
Fax: 303.440.1001  
www.minitech.com/medivators

LOT: 668397  
MFD: 03/2011  
EXP: 03/2012

**PART B Rapicide PA** Catalog No. ML02-0117  
**HIGH LEVEL DISINFECTANT**  
USE ONLY WITH ADVANTAGE™ PLUS OR DSD EDGE™ ENDOSCOPE REPROCESSING SYSTEMS

**KEEP OUT OF REACH OF CHILDREN DANGER**

**NET CONTENTS:**  
5.0 Liters  
(1.32 Gallons)

**Active Ingredients:**  
Tetraammonium phosphate 4.32%  
Inert Ingredients 95.68%  
Total 100.0%

MEDIVATORS®  
Manufactured in the USA by  
Minitech Corporation  
10000 W. 15th St., Suite 100  
Denver, CO 80202  
Tel: 303.440.1000  
Fax: 303.440.1001  
www.minitech.com/medivators

LOT: 668400  
MFD: 03/2011  
EXP: 03/2012

**Rapicide PA**  
HIGH LEVEL DISINFECTANT & STERILANT TEST STRIPS  
Primary Test Strips for Rapicide PA  
4.0 Liters of Rapicide PA  
Rapicide PA Ready-to-Use High Level Disinfectant Solution  
100 - 2000 ppm  
100 Strips  
ML02-0117

 **MEDIVATORS**  
A Cantel Medical Company

## RAPICIDE® PA High-Level Disinfectant

RAPICIDE PA is a single-use, peracetic acid-based high-level disinfectant that is environmentally responsible and requires no neutralization. It is designed for use in the MEDIVATORS® ADVANTAGE PLUS®, MEDIVATORS DSD EDGE® or MEDIVATORS RAPIDAER® Endoscope Reprocessors

- 5-minute contact time
- Requires only 1 rinse, reducing the total cycle time
- Proven endoscope compatibility – no increased wear on endoscope materials when used as recommended
- Effectively kills TB, hepatitis viruses C, diff\*, VRE\* and MRSA\*

- Single-use, aldehyde-free chemistry
- Environmentally responsible – breaks down to water and vinegar without neutralization issues
- Reduced worker exposure – no pouring, activating or mixing of HLD required
- Minimum recommended concentration (MRC) is 850ppm of PAA



\*Technical data available upon request

## RAPICIDE® PA Ready-to-Use High-Level Disinfectant and Sterilant

RAPICIDE PA Ready-to-Use is a peracetic acid-based reusable chemistry. It is designed for use on flexible endoscopes in automated endoscope reprocessors and manual applications

- High-level disinfects in 5 minutes at room temperature (20°C)
- Sterilant in 10 minutes at room temperature (20°C)
- Requires only 1 rinse, reducing the total cycle time
- Reusable Aldehyde-free chemistry
- Environmentally responsible peracetic acid (PAA formula)

- Proven endoscope compatibility – no increased wear on endoscope materials when used as directed
- Up to 14-day reuse life
- Ready-to-use: pour Part A solution into the bottle of Part B
- Minimum recommended concentration (MRC) is 1000ppm of PAA



## RAPICIDE® PA Spill Kits

Quick and easy way to neutralize and clean up spills at your facility.

Spill Kits include:

- NEUTRA-OX™ Sodium Bicarbonate neutralizer
- MSDS information
- Personal protective equipment
- Items to safely clean up spills or leaks



## RAPICIDE® PA Test Strips

RAPICIDE PA test strips are used to measure the concentration of RAPICIDE PA High-Level Disinfectant

- Color chart to confirm concentration is shown on test strip bottle
- 1-second dip; 30-second read time
- Large, easy-to-read indicator pad



### ORDERING INFORMATION

Part number	Description
ML02-0115	RAPICIDE PA (single-use) HLD
ML02-0125	RAPICIDE PA RTU (reusable) HLD

### ORDERING INFORMATION

Part number	Description
78400-974	Spill Kit for RAPICIDE PA Chemistries
DNS-1006	NEUTRA-OX Sodium Bicarbonate Neutralizer
ML02-0129	Test Strips for RAPICIDE PA Chemistry

MEDIVATORS® ADVANTAGE PLUS®, DSD EDGE® and RAPICIDE® are registered trademarks of Medivators Inc. RAPIDAER is a registered trademark of Cantel Medical (UK) Limited.



Medivators Inc.  
14605 28th Avenue North  
Minneapolis, MN  
55447-4822 USA  
Tel: +1.763.553.3300  
Fax: +1.763.553.3387

Medivators BV  
Sourethweg 11  
6422 PC Heerlen  
The Netherlands  
Tel: +31.45.5.471.471  
Fax: +31.45.5.429.595

[www.medivators.com](http://www.medivators.com)

Cantel Medical (UK) Limited  
Wolseley House  
Staffordshire Technology Park  
Beaconside, Stafford ST18 0GA  
United Kingdom  
Tel: +44 (0) 1785.782420  
Fax: +44 (0) 1785.782421

Cantel Medical Asia/Pacific Pte. Ltd.  
1A International Business Park  
#05-01 Singapore 609933  
Tel: +65.6227.9698  
Fax: +65.6225.6848

Medivators Inc. Beijing Representative Office  
Room 1801, Floor 18th, Tower A,  
Beijing Marriott Hotel, Office Building  
No. 7, Jianguomen South Avenue,  
Dongcheng District, Beijing 100005 China  
Tel: +86 10 65204039