MEDIVATORS® RAPICIDE® Reusable High Level Disinfectant Family MEDIVATORS



MEDIVATORS® RAPICIDE®

Reusable High Level Disinfectant Family

RAPICIDE® PA Ready-to-Use High-Level Disinfectant and Sterilant

RAPICIDE PA Ready-to-Use is a peracetic acid-based reusable chemistry. It is designed for use on flexible endoscopes in automated endoscope reprocessors and manual applications.

- · High-level disinfects in 5 minutes at room temperature (20°C)
- Sterilant in 10 minutes at room temperature (20°C)
- Aldehyde-free
- Up to 14-day use life
- Requires only 1 rinse, reducing the total cycle time
- Ready-to-use: pour Part A solution into the bottle of Part B
- Minimum recommended concentration (MRC) is 1000ppm of PAA

RAPICIDE® PA Test Strips

RAPICIDE PA test strips are used tomeasure the concentration of RAPICIDE PA High-Level Disinfectant.

- · Color chart to confirm concentration is shown on test strip bottle
- 1-second dip; 30-second read time
- · Large, easy-to-read indicator pad



RAPICIDE® High-Level Disinfectants... synonymous with speed and compatibility

RAPICIDE® High-Level Disinfectant and Sterilant

RAPICIDE glutaraldehyde is a fast-acting, non-staining reusable disinfectant. Its long use life and short contact time provides fast endoscope turnaround and improved efficiency.

- 5-minute contact time
- Used at 35°C
- · Effectively kills TB, hepatitis viruses and C. diff*
- Up to 28-day use life
- Improved efficiency; requires only 2 rinses
- · Ready to use, no mixing or activation required
- · Active chemical is 2.5% glutaraldehyde
- Minimum recommended concentration (MRC) is 1.5% glutaraldehyde

RAPICIDE® Test Strips

RAPICIDE test strips are used to measure the concentration of RAPICIDE High-Level Disinfectant and Sterilant.

- Color chart to confirm concentration is shown on test strip bottle
- · 3-second dip time; 75-second read time
- · Large, easy-to-read indicator pad





RAPICIDE® OPA/28 **High-Level Disinfectant**

RAPICIDE OPA/28 is an ortho-Phthalaldehyde (OPA) based high-level disinfectant that features a reuse period twice the length of competitors. With the same contact time and temperature for manual disinfection as in an automated endoscope reprocessor, RAPICIDE OPA/28 provides fast turnaround time for endoscopes.

- · 5-minute contact time
- Used at room temperature (20°C)
- Effectively kills TB, VRE*, and MRSA*
- Up to 28 day use-life
- Available for manual disinfection
- Ready-to-use, no mixing or activation required
- Active chemical is 0.575% OPA
- Minimum recommended concentration (MRC) is 0.35% OPA

RAPICIDE® OPA/28 Test Strips

RAPICIDE OPA/28 test strips are used to measure the concentration of RAPICIDE OPA/28 High-Level Disinfectant.

· Color chart to confirm concentration is shown on test strip bottle

 3-second dip; 90-second read time

· Large, easy-to-read indicator pad



RAPICIDE™ Spill Kits

The RAPICIDE Spill Kits are available for use with the MEDIVATORS family of RAPICIDE® products. The spill kits allow customers a quick and easy way to neutralize spills at their facility. The kit includes the appropriate neutralizer and the Personal Protective Equipment (PPE) necessary to safely clean up a spill or leak.



The RAPICIDE Spill Kit can be used for RAPICIDE® Glut High-Level Disinfectant and RAPICIDE OPA/28 High-Level Disinfectant. It includes:

- NEUTRA-HYDE® glycine neutralizer
- NEUTRA-HYDE SDS
- RAPICIDE glut SDS
- RAPICIDE OPA/28 SDS
- Bouffant cap

- · Nitrile gloves
- Impervious gown
- Shoe covers
- Safety goggles with face shield
- Large absorbent mats



The RAPICIDE PA Spill Kit can be used to clean up RAPICIDE PA High-Level Disinfectant. It includes:

- NEUTRA-Ox sodium bicarbonate neutralizer
- NEUTRA-Ox MSDS
- RAPICIDE PA MSDS
- Bouffant capNitrile gloves
- Impervious gown
- Shoe covers
- Safety goggles with face shield
- Mixing knife

ORDERING INFORMATION

RAPICIDE PA Ready-to-Use High-Level Disinfectant

Part number	Description
ML02-0125	RAPICIDE PA RTU High-Level Disinfectant (2 225mL Part A and 2 5 Liter Part B/case)
ML02-0129	RAPICIDE PA RTU Test Strips (2 bottles/pack, 100 strips/ bottle)

RAPICIDE Disinfectant and Sterilant

Part number	Description
ML02-0059	RAPICIDE High-Level Disinfectant and Sterilant (4 one-gallon bottles/case)
ML02-0120	RAPICIDE Test Strips (2 bottles/pack, 60 strips/ bottle)

RAPICIDE OPA/28 High-Level Disinfectant

Part number	Description
ML02-0132	RAPICIDE OPA/28 High-Level Disinfectant (4 one-gallon bottles/case)
ML02-0136	RAPICIDE OPA/28 Test Strips (2 bottles/pack, 50 strips/ bottle)

High-Level Disinfectant Spill Kits

Part number	Description
78400-973	For RAPICIDE Glut and RAPICIDE OPA/28 (1 Kit)
78400-974	For RAPICIDE PA (1 Kit)
DNS-1002	NEUTRA-HYDE Neutralizer
DNS-1006	NEUTRA-Ox Neutralizer

MEDIVATORS®, RAPICIDE® and NEUTRA-HYDE® are registered trademarks of Medivators Inc.



MEDIVATORS Inc. Headquarters 14605 28th Avenue North Minneapolis, MN 55447-4822 USA Tel: +1.800.444.4729 Fax: +1.763.553.3387

MEDIVATORS BV Sourethweg 11 6422 PC Heerlen The Netherlands Tel: +31.45.5.471.471 Fax: +31.45.5.429.695 MEDIVATORS Asia/Pacific Pte. Ltd.
1 A International Business Park
#05-01 Singapore 609933
Tel: +65.6227.9698
Fax: +65.6225.6848

MEDIVATORS Beijing Representative Office Room 708, 7th Floor Kaiheng Center, Block B No. 2 Chaoyangmennei Street Dongcheng District, Beijing China 100010 Tel: +8610.6567.8446 Fax: +8610.6567.8445