

Active Ingredient:
Glycine

MEDIVATORS®
NEUTRA-HYDE®
Glycine Neutralizer

Net Contents:
10 Oz. Bottle

**Environmentally responsible neutralizer for
glutaraldehyde (Glut) and ortho-phthalaldehyde (OPA)**

PRECAUTIONARY STATEMENTS:



This product is NOT to be ingested or brought into contact with the skin and/or eyes. If contact occurs, flush with cold water for fifteen (15) minutes. If ingested, contact a physician for medical attention. Do NOT put neutralizer into an automated endoscope reprocessor (AER) for neutralization. Used disinfectant should be transferred into a separate tank for neutralization. When used in an open soak container, sprinkle evenly into solution and stir until all powder is dissolved. Dispose of all waste products in accordance with all federal, state and local regulations. Keep away from children. Review and understand manufacturer's MSDS prior to use.

**Reorder #
DNS-1002**



Manufactured for:
MEDIVATORS
14605 28th Avenue North
Minneapolis, MN 55447 USA
Toll Free: +1.800.444.4729

Use one 10 ounce bottle of NEUTRA-Hyde to neutralize disinfectant as indicated in the chart below.

Disinfectant	# Gallons	Deactivation Time*
Glutaraldehyde	10	5 minutes
OPA	5	15 minutes

* Color change will occur during deactivation.

Sprinkle evenly into solution and replace lid. If using for a spill, sprinkle liberally over spill and refer to your spill response plan for further instructions.

67199-255/C