



PRODUCT BULLETIN

Distribution:	
International Distributors	√
International Management	√
US Clinical Specialists	√
US Customer Service	√
US Field Sales	√
US Technical and Field Service	√

Date: **September 20, 2010**

RE: Rapicide PA Test Strips Quality Control Procedure and RO water

Medivators has received inquiries regarding a source for the Reverse Osmosis (RO) water that is specified to perform the Quality Control procedure listed in the Rapicide PA Test Strips Directions for Use.

In this Quality Control procedure, it states that Rapicide PA Part A needs to be diluted with purified RO water. Most facilities do not have RO water system readily available. If your customer would like to follow this procedure and does not have a RO water system, RO water can be purchased at most grocery or convenience stores -- wherever bottled water is sold. The labeling on most brands indicates the purified water in the bottle is indeed RO water. This bottled water can then be used for the Quality Control tests.

Per the Rapicide PA Directions for Use, **Minntech certifies that the Rapicide PA Test Strips are quality control tested before shipping to customers.** A Certificate of Analysis is available on the website at <http://www.minntech.com/applications/DocumentLibraryManager/index.asp> by entering the lot number of the test strips. **Please note: The Quality Control procedure is optional for a customer to perform.**

If you have any questions regarding this Product Bulletin please contact the Medivators Customer Support group at 1-800-444-4729.

Endoscope reprocessing the way it should be

MEDIVATORS

Minntech Corporation, 14605 28th Avenue North, Minneapolis, MN 55447-4822

763.553.3300, fax 763.553.3387, www.minntech.com