# SCOPE BUDDY™ PLUS

## Endoscope Flushing Aid | Dosing Verification and Calibration Procedure

To ensure accurate dosing of detergent with the SCOPE BUDDY™ PLUS Endoscope Flushing Aid, Dosing Verification must be performed daily. If Dosing Verification fails, Dosing Calibration procedure must be performed and Verification repeated. Record all results using the Dosing Verification & Calibration Log. Contact Medivators customer support at 1-800-444-4729 or your local distributor (if located outside the U.S.) for further assistance.

### DAILY Perform SCOPE BUDDY PLUS Endoscope Flushing Aid Dosing Verification Procedure:

1. Ensure Pump Tube is installed and empty (air purge).
2. Ensure Detergent In Tube is installed and primed.
3. Direct the luer output of Pump Tube into a graduated cylinder.
4. Enter **MANUAL MODE** from the Home Screen.
5. Scan or enter User ID; for Endoscope ID and if applicable, Patient ID, use keypad to enter "1,2,3".
6. Select 'Dosing'.
7. Ensure **Sink**  icon is selected.
8. Ensure 20 mL is volume displayed. If not, select **keypad**  icon, press **backspace**  icon to clear the field and then use the number pad to enter 20 mL. Press **OK** to confirm.
9. Press **Play** to dose detergent into the graduated cylinder.
10. Remove Pump Tube from graduated cylinder and read dispensed volume.
  - If volume is within the range of 18 - 22mL:
    - Dosing is accurate and verification is complete.
    - Record a "Pass" result on the "SCOPE BUDDY PLUS Endoscope Flushing Aid Dosing Verification & Calibration Log".
  - If volume is outside the 18 - 22mL range:
    - Record a "Fail" result on the "SCOPE BUDDY PLUS Endoscope Flushing Aid Dosing Verification & Calibration Log".
    - Perform the Dosing Calibration procedure.
    - Perform Dosing Verification Procedure again.

### When Dosing Verification fails, perform Dosing Calibration by:

1. Return to the Home screen by pressing the **Home**  icon.
2. Press the  **Down Arrow**.
3. Select the **Settings**  icon.
4. Enter the SCOPE BUDDY PLUS Flushing Aid Settings password using the keypad.
5. Use the arrows to navigate to the Dosing Calibration menu.
6. Place open luer end of Pump Tube into empty sink.
7. Press **Play** to begin initializing.
8. Direct the luer output of the Pump Tube into a graduated cylinder.
9. On the Dosing Calibration (2 of 3) screen, press **Play** to begin the calibration.
10. Press the **keypad**  icon at the bottom to enter the measured volume.
11. The screen will display the entered volume. To change the volume, press **backspace**  icon and re-enter the volume.
12. Do **not** empty the graduated cylinder after the first step, dosing calibration is not complete.
13. Direct the luer output of the Pump Tube into the **SAME** graduated cylinder.
14. On the Dosing Calibration (3 of 3) screen, press **Play** to begin the calibration.
15. Press the **keypad**  icon at the bottom to enter the total measured volume.
16. Press **OK**, when calibration is complete.
17. Repeat Dosing Verification. If Dosing Verification fails again, contact Medivators Technical Support.
18. Document on Dosing Verification & Calibration Log.

www.medivators.com • 1-800-444-4729

# SCOPE BUDDY™ PLUS

## Endoscope Flushing Aid | Dosing Verification and Calibration Procedure

MONTH / YEAR: \_\_\_\_\_ SERIAL #: \_\_\_\_\_

Circle Results and Sign

Date	Dosing Verification		If Verification fails, Calibration done?		Dosing Re-Verification		Signature
	PASS	FAIL	YES	NO	PASS	FAIL	
1	PASS	FAIL	YES	NO	PASS	FAIL	
2	PASS	FAIL	YES	NO	PASS	FAIL	
3	PASS	FAIL	YES	NO	PASS	FAIL	
4	PASS	FAIL	YES	NO	PASS	FAIL	
5	PASS	FAIL	YES	NO	PASS	FAIL	
6	PASS	FAIL	YES	NO	PASS	FAIL	
7	PASS	FAIL	YES	NO	PASS	FAIL	
8	PASS	FAIL	YES	NO	PASS	FAIL	
9	PASS	FAIL	YES	NO	PASS	FAIL	
10	PASS	FAIL	YES	NO	PASS	FAIL	
11	PASS	FAIL	YES	NO	PASS	FAIL	
12	PASS	FAIL	YES	NO	PASS	FAIL	
13	PASS	FAIL	YES	NO	PASS	FAIL	
14	PASS	FAIL	YES	NO	PASS	FAIL	
15	PASS	FAIL	YES	NO	PASS	FAIL	
16	PASS	FAIL	YES	NO	PASS	FAIL	

Date	Dosing Verification		If Verification fails, Calibration done?		Dosing Re-Verification		Signature
	PASS	FAIL	YES	NO	PASS	FAIL	
17	PASS	FAIL	YES	NO	PASS	FAIL	
18	PASS	FAIL	YES	NO	PASS	FAIL	
19	PASS	FAIL	YES	NO	PASS	FAIL	
20	PASS	FAIL	YES	NO	PASS	FAIL	
21	PASS	FAIL	YES	NO	PASS	FAIL	
22	PASS	FAIL	YES	NO	PASS	FAIL	
23	PASS	FAIL	YES	NO	PASS	FAIL	
24	PASS	FAIL	YES	NO	PASS	FAIL	
25	PASS	FAIL	YES	NO	PASS	FAIL	
26	PASS	FAIL	YES	NO	PASS	FAIL	
27	PASS	FAIL	YES	NO	PASS	FAIL	
28	PASS	FAIL	YES	NO	PASS	FAIL	
29	PASS	FAIL	YES	NO	PASS	FAIL	
30	PASS	FAIL	YES	NO	PASS	FAIL	
31	PASS	FAIL	YES	NO	PASS	FAIL	

If the Dosing Re-Verification fails, contact Medivators Technical Support : Toll Free: 1-800-444-4729 • [www.medivators.com](http://www.medivators.com)

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