

Efficacy of RAPICIDE® High Level Disinfectant, RAPICIDE® PA High Level Disinfectant and RAPICIDE® OPA/28 High Level Disinfectant for Endoscope Reprocessing

As part of Cantel Medivators’ extensive development to deliver fast and effective High-Level Disinfection solutions, rigorous testing has been conducted to evaluate the efficacy and microbiological consideration of RAPICIDE HLD, RAPICIDE PA HLD and RAPICIDE OPA/28 HLD. All tests demonstrated sporicidal, bactericidal, fungicidal, tuberculocidal, and virucidal efficacy.

Tests	Organism	Results		
		RAPICIDE® HLD	RAPICIDE® PA HLD	RAPICIDE® OPA/28 HLD
		5 minutes at 35°C at MRC concentration equal to 1.5% Glutaraldehyde	5 minutes at 30°C at MRC concentration equal to 850ppm PAA	5 minutes at 25°C at MRC concentration equal to 0.35% OPA
Sporicidal	<i>Clostridium sporogenes</i>	Complete Inactivation in 5 hours	Complete Inactivation in 5 hours	Complete Inactivation in 32 hours
Sporicidal	<i>Bacillus subtilis</i>	Complete Inactivation in 5 hours	Complete Inactivation in 5 hours	Complete Inactivation in 32 hours
Tuberculocidal	<i>Mycobacterium bovis (var. BCG)</i>	>6 log Reduction	>6 log Reduction	>6 log Reduction
Virucidal	Poliovirus	Complete Inactivation	Complete Inactivation	Complete Inactivation
Virucidal	Adenovirus	Complete Inactivation	> 6 log Reduction at 20°C	Equivalent Efficacy to Herpes Simplex Virus
Virucidal	Herpes Simplex Virus	Complete Inactivation	Complete Inactivation	Complete Inactivation
Fungicidal	<i>Candida albicans</i>	>6 log Reduction	Equivalent Efficacy to <i>Trichophyton mentagrophytes</i>	
Fungicidal	<i>Aspergillus niger</i>	>6 log Reduction		
Fungicidal	<i>Trichophyton mentagrophytes</i>	>6 log Reduction	>6 log Reduction	>6 log Reduction
Bactericidal	<i>Pseudomonas aeruginosa</i>	>6 log Reduction	>6 log Reduction	>6 log Reduction
Bactericidal	<i>Staphylococcus aureus</i>	>6 log Reduction	>6 log Reduction	>6 log Reduction
Bactericidal	<i>Salmonella enterica</i>	>6 log Reduction	>6 log Reduction	>6 log Reduction
Simulated Use (no pre or post rinse)	<i>Mycobacterium terrae</i>	>6 log Reduction	>6 log Reduction	>6 log Reduction
In-Use	GI Tract Normal Flora	Total Kill	Total Kill	Total Kill

- Notes :
- 1) Inactivation
 - Implies reduced to state of non-living for viruses (which has no formal category as either living or non-living)
 - Implies rendered harmless for spores
 - 2) MRC
 - Minimum Recommended Concentration

The Cantel Medivators family of high-level disinfectants is also effective against CRE, VRE and MRSA. The germicides have effected a >99.9999% or >6.04-log reduction in the number of CRE in 1 minute at the following conditions.

Tests	Organism	Results		
		RAPICIDE® HLD	RAPICIDE® PA HLD	RAPICIDE® OPA/28 HLD
		1 minute at 35°C at nominal concentration equal to 2.5% Glutaraldehyde	1 minute at 30°C at nominal concentration equal to 1300ppm PAA	1 minute at 25°C at nominal concentration equal to 0.575% OPA
Bactericidal	Carbapenem-resistant Enterobacteriaceae (CRE)	>6 log Reduction	>6 log Reduction	>6 log Reduction
Bactericidal	Vancomycin-resistant <i>Enterococcus</i> (VRE)	RAPICIDE PA HLD and RAPICIDE OPA/28 HLD are more suited for such application	>5.6 log reduction	>6.49 log reduction *5 Minutes at 20°C
Bactericidal	Methicillin-resistant <i>Staphylococcus aureus</i> (MRSA)		>5.7 log Reduction	>6.74 log reduction *5 Minutes at 20°C

Conclusion

To meet the needs of healthcare facilities, Cantel Medivators offers a variety of fast and cost-effective high-level disinfectants that provide superior cleaning with the speed and compatibility you have come to recognize from Medivators.

RAPICIDE® is a registered trademark of Medivators Inc.

References

1. Kelleen Gutzmann, "Chemical Disinfectants and Antiseptics – Virucidal Quantitative Suspension Test for Chemical Disinfectants and Antiseptics Used in Human Medicine – Test Method and Requirements (Phase 2 Step 1)", ATS Labs, July 19, 2010.
2. Cantel Medivators. "Rapicide Safety, Efficacy and Microbiological Considerations - ML06-0028 Rev F", 2011, accessed from <http://www.medivators.com/sites/default/files/minntech/documents/ML06-0028%20REV%20F.pdf>.
3. Cantel Medivators. "Rapicide PA Disinfectant Safety, Efficacy and Microbiological Considerations - 50096-959 Rev D", 2011, accessed from <http://www.medivators.com/sites/default/files/minntech/documents/50096-959%20REV%20D.pdf>.
4. Cantel Medivators. "Rapicide OPA/28 Safety, Efficacy and Microbiological Considerations - 50098-052 Rev B", 2014, accessed from <http://www.medivators.com/sites/default/files/minntech/documents/50098-052%20REV%20B.pdf>.
5. Cantel Medivators. "Carbapenem-resistant Enterobacteriaceae (CRE) - 50098-443 Rev A", January 15, 2013, accessed from <http://www.medivators.com/sites/default/files/minntech/documents/50098-443%20REV%20A.PDF>.
6. Cantel Medivators. "Rapicide High Level Disinfectant Efficacy against CRE - 50098-712 Rev A", February 20, 2015, accessed from <http://www.medivators.com/sites/default/files/minntech/documents/50098-712%20REV%20A.pdf>.
7. Cantel Medivators. "Efficacy of RAPICIDE High-Level Disinfectant, RAPICIDE PA and RAPICIDE OPA/28 against Carbapenem-resistant Enterobacteriaceae (CRE) - 50098-787 Rev A", April 7, 2015, accessed from <http://www.medivators.com/sites/default/files/minntech/documents/50098-787%20REV%20A.pdf>.
8. Cantel Medivators. "Efficacy of Rapicide PA High-Level Disinfectant against Bacterial Endospores - 50097-174 Rev B", February 21, 2011, accessed from <http://www.medivators.com/sites/default/files/minntech/products/documents/50097-174RevB.pdf>.
9. Cantel Medivators. "Rapicide PA Efficacy against VRE - 50097-032 Rev D", July 29, 2011, accessed from <http://www.medivators.com/sites/default/files/minntech/documents/50097-032%20Rev%20D.pdf>.
10. Cantel Medivators. "Rapicide PA Efficacy against MRSA - 50097-033 Rev D", July 29, 2011, accessed from <http://www.medivators.com/sites/default/files/minntech/documents/50097-033%20Rev%20D.pdf>.
11. Cantel Medivators. "Rapicide OPA/28 Efficacy against Tuberculosis - 50098-055 Rev A", January 22, 2013, accessed from <http://www.medivators.com/sites/default/files/minntech/documents/50098-055%20Rev%20A.pdf>.
12. Cantel Medivators. "Rapicide OPA/28 Efficacy against VRE - 50098-054 Rev A", January 22, 2013, accessed from <http://www.medivators.com/sites/default/files/minntech/documents/50098-054%20Rev%20A.pdf>.
13. Cantel Medivators. "Rapicide OPA/28 Efficacy against MRSA - 50098-053 Rev A", January 22, 2013, accessed from <http://www.medivators.com/sites/default/files/minntech/documents/50098-053%20REV%20A.pdf>.
14. British Standards Institute (BSI). EN 14347. *Chemical disinfectants and antiseptics. Basic sporicidal activity. Test method and requirements (phase 1, step 1)*. BSI, London, United Kingdom (2005).
15. British Standards Institute (BSI). EN 14348. *Chemical disinfectants and antiseptics. Quantitative suspension test for the evaluation of mycobactericidal activity of chemical disinfectants in the medical area including instrument disinfectants. Test methods and requirements (phase 2, step 1)*. BSI, London, United Kingdom (2005).

16. British Standards Institute (BSI). EN 14476. *Chemical disinfectants and antiseptics. Virucidal quantitative suspension test for chemical disinfectants and antiseptics used in human medicine. Test method and requirements (phase 2, step 1)*. BSI, London, United Kingdom (2005).
17. British Standards Institute (BSI). EN 14562. *Chemical disinfectants and antiseptics. Quantitative carrier test for the evaluation of fungicidal or yeasticidal activity for instruments used in the medical area. Test method and requirements (phase 2, step 2)*. BSI, London, United Kingdom (2006).
18. British Standards Institute (BSI). EN 14561. *Chemical disinfectants and antiseptics. Quantitative carrier test for the evaluation of bactericidal activity for instruments used in the medical area. Test method and requirements (phase 2, step 2)*. BSI, London, United Kingdom (2006).
19. AOAC International. *Official Methods of Analysis Method 966.04 – Sporocidal Activity of Disinfectants*. Gaithersbury, Maryland, United States (2005).
20. AOAC International. *Official Methods of Analysis Method 955.17 – Fungicidal Activity of Disinfectants*. Gaithersbury, Maryland, United States (2005).
21. AOAC International. *Official Methods of Analysis Method 965.12 – Tuberculocidal Activity of Disinfectants*. Gaithersbury, Maryland, United States (2005).
22. AOAC International. *Official Methods of Analysis Method 964.02, 955.14 and 955.15 – Use Dilution*. Gaithersbury, Maryland, United States (2005).
23. International Organization for Standardization (ISO). *ISO 15883-1:2006: Washer-Disinfectors – Part 1 General Requirements, Terms and Definitions and Tests*. Geneva, Switzerland (2006).
24. Food and Drug Administration et al. *Guidance for Industry and FDA Reviewers: Content and Format of Premarket Notification [510(k)] Submissions for Liquid Chemical Sterilants/High Level Disinfectants*. United States (2000).