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RAPICIDE™ is a trademark of Medivators Inc.

Manufactured in the USA by:

Medivators Inc.
 14605 28th Avenue North
 Minneapolis, MN 55447 USA
 Toll Free: +1.800.444.4729

www.medivators.com

4x1 Gallon bottles per case
 © 2019 Medivators Inc.

DIRECTIONS FOR USE: It is a violation of Federal (USA) Law to use this product in a manner inconsistent with its labeling. Read the Direction for Use for more detailed information. For Cleaning / Decontamination, Sterilization, High Level Disinfection, and Reuse period information read the Directions for Use.

PRECAUTIONARY STATEMENTS:


HAZARD TO HUMANS AND DOMESTIC ANIMALS

Direct contact is corrosive to exposed tissue, causing eye and skin damage. Do not get in eyes, on skin or clothing, and avoid breathing the fumes. Avoid contamination of food. Wear protective eye covers and Nitrile gloves when handling or pouring.

STATEMENT OF PRACTICAL TREATMENT: In case of contact, immediately flush eyes or skin with copious amounts of water for at least fifteen (15) minutes. Get medical attention for eye contact. Harmful if swallowed. Drink large quantities of water and call a physician immediately. **NOTE TO PHYSICIAN:** Probable mucosal damage may contraindicate gastric lavage.

USER PROFICIENCY: The user should be adequately trained in the decontamination and disinfection or sterilization of medical devices and handling of toxic substances such as liquid chemical germicides.

STORAGE AND DISPOSAL: Store at controlled temperature 15°C-25°C (59°F-77°F). Once opened, the contents of container should be used within 28 days. Discard residual solution according to local, provincial, or federal regulations. Do not reuse empty container. Triple rinse with water and dispose in an incinerator or landfill approved for pesticide containers. For additional information regarding the safety and effectiveness of RAPICIDE™ High-Level Disinfectant and Sterilant call 1-800-328-3340.

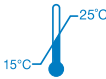LOT : _____
 MFD : _____
 EXP : _____

Distributed in Canada by:
 Cantel (Canada) Inc.
 6-88B Beaver Creek Rd E
 Richmond Hill, ON L4B 4W2
 Phone: 844.348.5636
 Fax: 844.348.5637

67198-254 Rev P