

MEDIVATORS™ **Rapicide™** OPA/28 TEST STRIPS

REF ML02-0136

Test Strips for the Concentration of RAPICIDE™ OPA/28 High-Level Disinfectant

50 Strips

IMPORTANT: Keep cap tight. Store at 15° - 30°C (59° - 86°F). Do not remove desiccant packet. This is a single use device discard after use. Lot Number and expiration date on bottom of bottle. Use within 6 months of opening. DATE OPENED _____

READ ENTIRE INSERT FOR ADDITIONAL INFORMATION.

DIRECTIONS:

- 1. Initiate timer and immerse entire indicator pad of test strip into the RAPICIDE OPA/28 solution for a full three (3) seconds.
2. Remove strip and touch long edge on a paper towel to remove excess OPA/28 solution.
3. Lay strip, pad side up, on a paper towel.
4. Compare indicator pad to color blocks at ninety (90) seconds and interpret result.

RESULTS:

Four empty boxes for results, with labels FAIL, FAIL, PASS, PASS below them.

FAIL FAIL PASS PASS

MEDIVATORS™ and RAPICIDE™ are trademarks of Medivators Inc.



A Cantel Medical Company



Medivators Inc. 14605 28th Avenue North Minneapolis, MN 55447, USA Toll Free: +1.800.444.4729

Medivators BV Sourthweg 11 6422PC Heerlen The Netherlands Tel: +31-46-5.471.471

Cantel Medical Asia/Pacific Pte Ltd 1A International Business Park #05-01 Singapore 609933 Tel: +65.6227.9898

www.medivators.com

MADE AND PRINTED IN THE USA

M174517750

11706

R3/18

67200-054/F