

SCOPE BUDDY™ PLUS

Endoscope Flushing Aid

USING SCOPE BUDDY™ PLUS ENDOSCOPE FLUSHING AID TO ASPIRATE OLYMPUS® DUAL CHANNEL ENDOSCOPES

CONNECTING THE ENDOSCOPE FOR ASPIRATION

- Place endoscope in sink
- Attach 24-Hour Multi-Use Hookup for OLYMPUS® Endoscopes to endoscope and close all pinch clips.
- Push blanking cap for OLYMPUS® Endoscopes onto the endoscope's air/water channels.
- Install the channel plug over valve openings, leaving both biopsy/instrument ports open.
- Attach pump tube to 24-Hour Multi-use Hookup for OLYMPUS® Endoscopes
- Remove the strainer from the end of the pump tube and set it on the counter.
- Hold end of the pump tube without the strainer over sink.

The **ASPIRATION** step runs the pump in reverse, drawing fluid through the biopsy channels.



ASPIRATION

- Select the desired aspiration time by using the up and down arrows. The minimum time recommended by Olympus is 30 seconds.
- Press PLAY to begin aspirating.

PREPARING THE ENDOSCOPE FOR THE NEXT STEP AFTER ASPIRATION

- Replace the strainer on the pump tube when aspiration is complete.
- Release pinch clips on the tubing attached to the endoscope. Leave the pinch clip closed on tubing if it is not connected to an endoscope port.
- Install the OLYMPUS® MAJ-420 (Medivators part # DSD-110-HU0159) over the endoscope's biopsy ports (Shown in picture) per Olympus IFU.

Installation of OLYMPUS® MAJ-420 over endoscope's biopsy ports after completing aspiration