

# CHEMFLEX™

## Refillable Dry Wipe System

CHEMFLEX™ Refillable Dry Wipe System has been validated with ACTRIL™ Cold Sterilant, other peracetic acid solutions, citric acid and bleach (1% and 10% concentrations). The wipe material is cellulose free and will not compromise the strength of the disinfectant like some cellulose-based (paper) products.

Once a disinfectant is added to the CHEMFLEX™ System, it remains stable for five days. The refillable, closed container provides an easy, accessible method to disinfect patient areas with less exposure to vapors and reduced risk of spills.

## FEATURES

- Validated with ACTRIL™ Cold Sterilant, a peracetic acid disinfectant
- Closed container system
- Reduces the risk of cross contamination
- Less exposure to vapors
- Cellulose and latex free
- Convenient pop-up container
- Low lint, large (7x10") and durable wipe material

## VALIDATION

Validation studies have been completed on PAA solutions. These studies examined the saturation levels, wipe integrity and chemical concentration over a 14-day period. Once a disinfectant is added to the CHEMFLEX™ System, it remains stable for five days. No testing is required during this time. Between days 6 and 14 you must test the disinfectant concentration daily using a test strip to ensure the disinfectant strength. Peracetic acid test strips are available for daily validation.

## SAFETY PROCEDURES

When using the dry wipes for the first time, write the name of the disinfectant used on the outside of the canister. When replacing dry wipes, refill using the same disinfectant indicated on the outside of the canister.

**NEVER PLACE A DIFFERENT TYPE OF DISINFECTANT IN THE CANISTER OTHER THAN WHAT IS INDICATED IN THE CHEMICAL FIELD ON THE OUTSIDE OF THE CANISTER.**

CHEMFLEX Refillable Dry Wipe System requires the operator to wear personal protective equipment (PPE) as defined in the labeling of the disinfectant used with the CHEMFLEX System.



## Ordering Information

UNITED STATES		
PART NUMBER	DESCRIPTION	QUANTITY
78400-287	CHEMFLEX™ Refillable Dry Wipe System	1 Canister & 6 Wipe Rolls
78400-309	ACTRIL™ Cold Sterilant	6 x 1 Liter Case
4001944	ACTRIL Cold Sterilant	4 x 5 Liter Case
78280-000	ACTRIL Cold Sterilant Test Strips	100 Test Strips/Pack

ACTRIL™ and CHEMFLEX™ are trademarks of Medivators, Inc.

[www.medivators.com](http://www.medivators.com)

TO PLACE AN ORDER

p: 1.800.444.4729 e: [custserv@medivators.com](mailto:custserv@medivators.com)

© 2020 Medivators Inc LIT-00205/A

