

Product Bulletin

For Distribution To:

- International Distributors
- International Management
- U.S. Clinical Specialists
- U.S. Customer Service
- U.S. Field Sales
- U.S. Technical & Field Service

Date: March 16, 2020

RE: CER OPTIMA™ Automated Endoscope Reprocessor (AER) Alcohol Injection Deactivation

The following procedure must only be performed when the unit is in an idle state and the daily Quality Assurance has been performed.

1. Select the Automatic Cycle in use, either FULL or HLD/RINSE (the image below shows the unit set to HLD/RINSE cycle).



2. Record the current alcohol setting and save for future reference. Possible options are 0, 30, 60 or 120.
3. Press the blue ALCOHOL button until the number after the ALC on the screen reads 0.
4. If both the HLD/RINSE and FULL cycles are used, change the alcohol setting for the other Automatic cycle to 0 in the same manner.
5. The alcohol bottle can be drained and rinsed. Indicate on the bottle or door of the alcohol department that the alcohol is not in use.
6. The AER is ready for operation.

To reactivate the alcohol injection:

1. Select the Automatic Cycle in use, either FULL or HLD/RINSE (the image above shows the unit set to HLD/RINSE cycle). Press the blue ALCOHOL button until the number after the ALC on the screen reads either 30, 60 or 120, based on prior settings.
2. If both the HLD/RINSE and FULL cycles are used, change the alcohol setting for FULL to the selected alcohol inject, either 30, 60 or 120.
3. The alcohol bottle should be rinsed and washed with a mild soap if required. Refill with alcohol.
4. The AER is ready for operation.

Contact Medivators Technical Support at 800-444-4729 with any questions.

CER OPTIMA™ is a trademark of Medivators Inc.



Medivators Inc.
14605 28th Avenue North
Minneapolis, MN
55447-4822 USA
Tel: +1.763.553.3300
Fax: +1.763.553.3387

www.cantelmedical.com

PCN-00042 Rev A