

Rapicide® PA READY-TO-USE

High-Level Disinfectant & Sterilant Test Strips


QUICK REFERENCE GUIDE*

* For use outside of USA and Canada

RAPICIDE® PA Ready-To-Use Test Strips provide a fast disinfectant reaction time and user-friendly color comparison chart. The indicator test strips are used for testing RAPICIDE® PA and RAPICIDE® PA Ready-to-Use High-Level Disinfectant and Sterilant, peracetic acid-based disinfectant.

TEST STRIP FEATURES

- Fast 30-second reaction time
- Definitive pass/fail color identification comparison chart
- 1000ppm minimum recommended concentration (MRC) level for RAPICIDE PA Ready-to-Use
- Shelf life is one (1) year unopened and four (4) months once opened
- Large, easy-to-read indicator pad

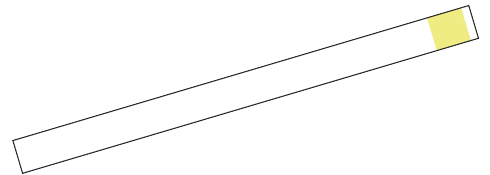

Test Strip Color (Untested)

PACKAGING

- Supplied 100 test strips per bottle, 2 bottles per package
- Medivators order number #ML02-0129

CUSTOMER SUPPORT

For questions or to place an order, please contact your Medivators or visit our website at www.cantelmedical.com

DIRECTIONS FOR USE

Read the entire RAPICIDE PA RTU Test Strips package instructions for use for important product information and proper usage instructions. Refer to the endoscope reprocessor user manual for instructions on how to take a sample of the disinfectant use solution in a reprocessor. It is recommended that the disinfectant solution be tested with each endoscope reprocessing cycle to verify disinfectant is above MRC.

- Open the RAPICIDE PA RTU Test Strips bottle and remove test strip. Close the bottle.
- Dip the test strip indicator pad into the disinfectant solution for exactly one (1) second and immediately remove.
- Wait exactly thirty (30) seconds and then compare the test strip color with the color chart located on the side of the test strip bottle. Test results showing PAA levels of 1000 ppm and higher are defined as Positive.

Test Strip Comparison Chart

